
en	Instruction For Use
bg	Инструкции за употреба
cs	Pokyny pro použití
da	Brugsanvisning
de	Gebrauchsanweisung
el	Οδηγίες χρήσης
es	Instrucciones de uso
et	Kasutusjuhised
fi	Käyttöohjeet
fr	Mode d'emploi
hu	Használati utasítás
it	Istruzioni per l'uso
lt	Naudojimo instrukcijos
lv	Lietošanas instrukcija
nl	Gebruiksaanwijzing
no	Bruksanvisning
pl	Instrukcja obsługi
pt	Instruções de utilização
ro	Instrucțiuni de utilizare
sk	Návod na použitie
sv	Bruksanvisning
tr	Kullanım talimatları

ARGON MEDICAL DEVICES, INC.
 1445 Flat Creek Road
 Athens, Texas 75751 USA
 Tel: 800-927-4669
 Tel: +1 903-675-9321
 www.argonmedical.com

EMERGO EUROPE
 Prinsessegracht 20
 2514 AP The Hague
 The Netherlands
 +31 70 345 8570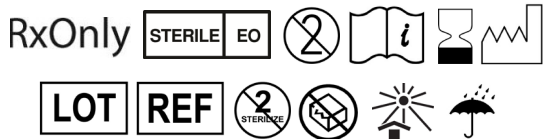

The symbols glossary is located electronically at www.argonmedical.com/symbols

IFU7552/0619C

Indication for use

The product is for biliary drainage

Duration

Up to 12 weeks

Warnings

- This device was designed, tested and manufactured for single use only. Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury. Do not reuse, reprocess or re-sterilize this device.
- Do not use if package appear open or if the expiry date has been exceeded.
- The product must only be used by qualified personnel who are familiar with the technique.

Preparation

- Advance the straightener carefully over the curvature of the catheter, while straightening the curvature with your fingers.
- Introduce the chosen stiffener (if plastic: activate the hydrophilic coating with saline) and fasten the stiffener onto the catheter (Luer Lock).
- Remove the straightener
- Activate the coating by saline.

Procedure

- Select and prepare the drainage site using standard technique.
- Insert an appropriate guidewire using standard technique.
- Dilate the tract, if necessary.
- Advance the catheter over the guidewire in fluoroscopy.
- When the catheter tip is inside the biliary tree, loosen the metal stiffener and replace it by the hydrophilic plastic stiffener. Advance the catheter over the guidewire as far as duodenum.
- Remove the stiffener.
- Verify correct position of the catheter using fluoroscopy.
- The radiopaque marker indicates the last side hole and must be positioned in the biliary passage.
- Remove the guidewire
- Connect the catheter to a drainage bag using a suitable connecting tube.

It is recommended affixing the catheter in a straight line and that any curvature be applied to the connecting tube.

It is recommended that the physician follow the hospital's standard of care procedures for drainage catheters.

Removal of catheter

- Disconnect the drainage bag connector tube from catheter.
- Pull the catheter out gently. If access is to be maintained, a straight floppy tip guidewire passed through the catheter will facilitate removal while maintaining access.

Storage

Store in a dark, dry, cool place. Avoid extended exposure to light.

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