en	Instruction For Use
bg	Инструкции за употреба
cs	Pokyny pro použití
da	Brugsanvisning
de	Gebrauchsanweisung
el	Οδηγίες χρήσης
es	Instrucciones de uso
et	Kasutusjuhised
fi	Käyttöohjeet
fr	Mode d'emploi
hu	Használati utasítás
it	Istruzioni per l'uso
lt	Naudojimo instrukcijos
lv	Lietošanas instrukcija
nl	Gebbruiksaanwijzing
no	Bruksanvisning
pl	Instrukcja obsługi
pt	Instruções de utilização
ro	Instrucțiuni de utilizare
sk	Návod na použitie
sv	Bruksanvisning
tr	Kullanım talimatları

**ARGON MEDICAL DEVICES, INC.**

1445 Flat Creek Road  
Athens, Texas 75751 USA  
Tel: 800-927-4669  
Tel: +1 903-675-9321  
www.argonmedical.com

**EMERGO EUROPE**

Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

RxOnly

The symbols glossary is located electronically at  
[www.argonmedical.com/symbols](http://www.argonmedical.com/symbols)

IFU7511/0617B

**en - English****Indication for use**

The product is for drainage of cysts, abscesses, haematomas, pleural exudates, ascites, gall bladders and nephrostomies.

**Duration**

Up to 12 weeks

**Warnings**

- This device was designed, tested and manufactured for single use only. Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury. Do not reuse, reprocess or re-sterilize this device.
- Do not use if package appear open or if the expiry date has been exceeded.
- The product must only be used by qualified personnel who are familiar with the technique.

**Preparation**

- Advance the straightener carefully over the curvature of the catheter, while straightening the curvature with your fingers.
- Introduce the chosen stiffener (if plastic: activate the hydrophilic coating with saline) and fasten the stiffener onto the catheter (Luer Lock).
- Remove the straightener
- Activate the coating by saline.

**Procedure**

- Advance the catheter over the guidewire that has been introduced into the patient using standard technique.
- When the catheter tip is inside the cavity, loosen the stiffener.
- Hold the stiffener tight and advance the catheter over the stiffener until the complete length of the pigtail is in the cavity.
- Remove the stiffener.
- Verify correct position of the catheter using fluoroscopy.
- Remove the guidewire
- Connect the catheter to a drainage bag using a suitable connecting tube.

It is recommended affixing the catheter in a straight line and that any curvature be applied to the connecting tube.

It is recommended that the physician follow the hospital's standard of care procedures for drainage catheters.

**Removal of catheter**

- Disconnect the drainage bag connector tube from catheter.
- Pull the catheter out gently. If access is to be maintained, a straight floppy tip guidewire passed through the catheter will facilitate removal while maintaining access.

**Storage**

Store in a dark, dry, cool place. Avoid extended exposure to light.

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