**ARGON MEDICAL DEVICES, INC.**  
 1445 Flat Creek Road, Athens, Texas 75751 USA  
 Tel: 800-927-4669;  
 Tel: +1 903-675-9321  
 www.argonmedical.com

**ENG:**

**Intended Use:** Intended for percutaneous drainage of abscess fluid, pleuracentesis, cysts, gall bladder, pleural emphyemas, lung abscesses and mediastinal collections.

**Contraindications:** Only qualified personnel who are familiar with the technique must use the product. During insertion avoid contact with bone, cartilage and scar tissue which can damage the catheter tip. **Please see Removal of Catheter.**

FRENCH	MAXIMUM GUIDEWIRE SIZE	CATHETER OD	CATHETER STIFFENER
6	0.035"	2.14 mm	METAL
7	0.035"	2.47 mm	METAL
8	0.038"	2.80 mm	METAL AND PLASTIC
10	0.038"	3.47 mm	METAL AND PLASTIC
12	0.038"	4.12 mm	METAL AND PLASTIC
14	0.038"	4.78 mm	METAL AND PLASTIC
16	0.038"	5.28 mm	METAL AND PLASTIC

**Preparation:**

1. Straighten the pigtail out with the pigtail straightener. Gently pull the suture to avoid unintentional looping of the suture.
2. Introduce and lock the metal stiffening cannula onto the catheter (Luer Lock). Introduce the stylet into the catheter, if using direct puncture insertion technique. Note that the stylet clicks into a notch on the cannula hub. Snap the Choice Lock onto the needle hub. Remove the pigtail straightener.
3. To activate the hydrophilic coating, wet the catheter in sterile water or saline. Keep the catheter wet during placement. Use a wet gauze pad to handle the catheter during placement if necessary. Do not wipe the catheter with dry gauze or any solvents as it may damage the coating.

**Procedure Using Direct Puncture Insertion Technique (Trocar/Stylet):**

1. Select and prepare the drainage site using conventional technique.
2. Perform the skin incision under local anesthesia.
3. Advance the catheter/cannula/trocar stylet into the cavity using fluoroscopy, CT or ultrasound guidance.
4. Trocar stylet is removed by releasing the Choice Lock from the cannula hub. Check that fluid is escaping or aspirate with a syringe. After the catheter is positioned, unlock the cannula from the catheter. (If desired a guidewire can be inserted at this point to aid in placement.) Holding the cannula stationary, advance the catheter to the desired location ensuring that the pigtail is in the cavity.
5. Under fluoroscopy, slowly remove the cannula while rotating the catheter counterclockwise. This movement will ensure correct position and cause the catheter to re-form the pigtail.
6. To lock the pigtail in its position:
  - Pull the thread gently.
  - Wind the thread around the slot and press the clip onto the slot. The thread must be stretched during the process.
  - Attach and tighten the female/male adapter onto the catheter to activate the valve in the catheter.
7. Connect the catheter to a drainage bag with connector tubing (Medical Device Technologies Catalog # DBAG600).
8. Check the position of the catheter using fluoroscopy.

**Procedure Using Seldinger Technique:**

1. Select and prepare the drainage site using conventional technique.
2. Using local anesthesia and Seldinger Technique, insert a guidewire into the cavity.
3. Progressively dilate the tract to 1 French size greater than the size of the catheter. This will ease introduction.
4. Wet the hydrophilic coated catheter to activate the coating (see "Preparation").
5. Under fluoroscopy, the catheter and cannula are advanced over the guidewire into the cavity. At the point of entry into the cavity, the cannula is unlocked and removed. The catheter is advanced over the guidewire into the cavity ensuring that the pigtail is in the cavity.
6. Under fluoroscopy, slowly remove the guidewire while rotating the catheter counterclockwise. This movement will ensure correct position and cause the catheter to re-form the pigtail.
7. To lock the pigtail in its position:
  - Pull the thread gently.
  - Wind the thread around the slot and press the clip onto the slot. The thread must be stretched during the process.
  - Attach and tighten the female/male adapter onto the catheter to activate the valve in the catheter.
8. Connect the catheter to a drainage bag with connector tubing (Medical Device Technologies Catalog # DBAG600).
9. Check the position of the catheter using fluoroscopy.

**Removal of Catheter:**

1. Disconnect the drainage bag connector tube from catheter.
2. Loosen the female/male adaptor from the standard luer hub at the catheter to deactivate the valve.
3. Remove the clip and unwind the suture. Check that both threads are loose and cut one thread in order to loosen the pigtail.
4. Pull the catheter out gently. If access is to be maintained, a straight floppy tip guidewire passed through the catheter will facilitate removal while maintaining access.

**Storage:** Store in a cool, dry area.

**ENG: SKATER® All-Purpose Drainage Catheter (with Locking Pigtail)**

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