
ARGON MEDICAL DEVICES, INC.
1445 Flat Creek Road
Athens, Texas 75751 USA
Tel: 800-927-4669;
Tel: +1 903-675-9321
www.argonmedical.com

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

The symbols glossary is located electronically at
www.argonmedical.com/symbols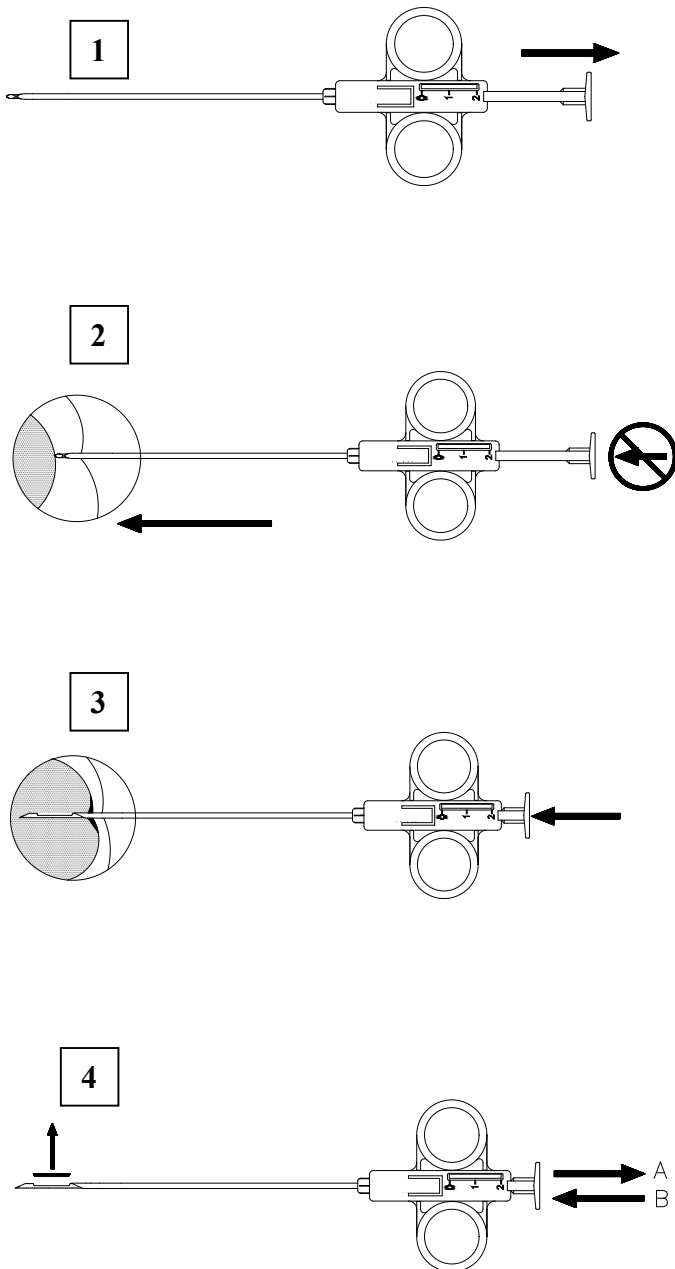

ENGLISH

Caution: Federal (USA) Law restricts this device to sale by or on the order of a physician.

SuperCore™ Biopsy Instrument

DEVICE DESCRIPTION: The SuperCore™ Biopsy Instrument is a semi-automatic biopsy instrument used to obtain soft tissue biopsies. It is composed of an echogenic spring loaded biopsy needle fitted into a plastic handle permitting single handed specimen collection. The position of the needle may be visualized by x-ray, CT or ultrasound. The needle has numerically ordered centimeter markings to facilitate precise depth placement. The adjustable instrument allows for a throw of 1.2cm or 2.2cm exposing the specimen notch either 19mm or 9.5mm, providing clinical flexibility.

INDICATIONS FOR USE: Disposable guillotine soft tissue instrument for obtaining percutaneous or surgical histological samples, (e.g. lung, liver, spleen, kidney, prostate, lymph nodes, breast, thyroid, and pancreas).

CONTRAINDICATIONS: These needles should be used by a physician familiar with the possible side effects, typical findings, limitations, indications and contraindications of soft tissue biopsy. Physician judgment is required when considering biopsy on patients with bleeding disorder, or receiving anti-coagulant medications.

PRECAUTIONS: This device was designed, tested and manufactured for single use only. Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury. Do not reuse, reprocess or re-sterilize this device. Handle in a manner that will prevent accidental puncture. This instrument can be used with an appropriate Argon Co-Axial Introducer Needle. Other manufacturer's co-axial introducer needles have not been tested for compatibility with the SuperCore™ Biopsy Instrument.

WARNINGS: These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

INSTRUCTIONS FOR USE:

- Optional use of co-axial introducer needle with biopsy instrument: Advance the co-axial introducer needle, using standard technique, to a position proximal to the lesion. Position the adjustable needle stop proximal to the skin. Remove the stylet from the co-axial introducer needle, leaving the cannula in position.
- Cock the SuperCore™ biopsy instrument by pulling back on the plunger until a firm click is felt at either the 1.2 or 2.2cm position. (User selectable.)
- With stylet fully retracted (specimen notch completely covered by cannula as in fig. 2, insert needle percutaneously or through the co-axial introducer needle proximal to area to be sampled. Warning: Be careful not to press plunger until needle is inserted to desired location. Verify proper location of needle.
- Move plunger with thumb to first stop to advance stylet exposing specimen notch (1.2 or 2.2cm). Move needle back and forth a few mm to fill specimen notch. Press plunger past stop with thumb, automatically triggering the cannula to close, trapping the specimen in the sample notch.
- Draw needle out. Pull back plunger to 1.2 or 2.2cm position, recharging the spring. Push plunger to first stop exposing the specimen notch. Remove tissue specimen from notch in stylet. Be careful not to push plunger past first stop during this step.

DISPOSAL: After use, this product may be a potential biohazard. Dispose in accordance with applicable laws and regulations.

STORAGE: Store at standard ambient temperature.

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SuperCore™ is a trademark of Argon Medical Devices, Inc.
©2013 Argon Medical Devices, Inc. All rights Reserved.