en - English

en	Instruction For Use
bg	Инструкции за употреба
cs	Pokyny pro použití
da	Brugsanvisning
de	Gebrauchsanweisung
el	Οδηγίες χρήσης
es	Instrucciones de uso
et	Kasutusjuhised
fi	Käyttöohjeet
fr	Mode d'emploi
hu	Használati utasítás
it	Istruzioni per l'uso
lt	Naudojimo instrukcijos
lv	Lietošanas instrukcija
nl	Gebruiksaanwijzing
no	Bruksanvisning
pl	Instrukcja obsługi
pt	Instruções de utilização
ro	Instrucțiuni de utilizare
sk	Návod na použitie
sv	Bruksanvisning
tr	Kullanım talimatları

Indication for use

The product is for drainage of cysts, gall bladders and nephrostomies.

Duration

Up to 8 weeks

Warnings

- This device was designed, tested and manufactured for single patient use only. Reuse or reprocessing of this device may lead to its failure and subsequent patient injury. Reprocessing and/or resterilization of this device may create the risk of contamination and patient infection. Do not reuse, reprocess or resterilize this device.
- Do not use if package appears open or if the expiry date has been exceeded.
- The product must only be used by qualified personnel who are familiar with the technique.

Preparation

- Advance the straightener carefully over the curvature of the catheter, while straightening the curvature with your fingers.
- The straightener now keeps the curvature straight allowing testing of the guidewire through the catheter before use.

Procedure

- Advance the catheter over the guidewire that has been introduced into the patient using standard technique.
- When the catheter is fully loaded onto the guidewire remove the straightener.
- Ensure that the complete length of the pigtail is in the cavity.
- Loosen the adapter and gently pull the thread turning the catheter clockwise to ensure correct position and to complete curvature.
- Fix the thread tightly by winding it around the slot in the adapter and finish up with pressing the clip onto the slot and twist off the handle from the clip.
- Verify correct position of the catheter using fluoroscopy.
- Remove the guidewire
- Connect the catheter to a drainage bag using a suitable connecting tube.

It is recommended affixing the catheter in a straight line and that any curvature be applied to the connecting tube.

It is recommended that the physician follow the hospital's standard of care procedures for drainage catheters.

Removal of catheter

- Disconnect the drainage bag connector tube from catheter.
- Loosen the female/male adapter from the standard luer hub at the catheter to deactivate the valve.
- Remove the clip and unwind the suture. Check that both threads are loose and cut one thread in order to loosen the pigtail.

Pull the catheter out gently. If access is to be maintained, a straight floppy tip guidewire passed through the catheter will facilitate removal while maintaining access.

ARGON MEDICAL DEVICES, INC.
 1445 Flat Creek Road
 Athens, Texas 75751 USA
 Tel: +1 (903) 675 9321
 Tel: +1 (800) 927 4669
 www.argonmedical.com

EMERGO EUROPE
 Prinsessegracht 20
 2514 AP The Hague
 The Netherlands
 +31 70 345 8570

The symbols glossary is located electronically at www.argonmedical.com/symbols

Caution

If the catheter is to be removed on another ward, we recommend that these guidance notes accompany the patient's case notes to ensure that the relevant personnel are aware of the presence of a locking catheter. It is also advisable to inform the patient.

Storage

Store in a dark, dry, cool place. Avoid extended exposure to light.

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