
ARGON MEDICAL DEVICES, INC.
 1445 Flat Creek Road, Athens, Texas 75751 USA
 Tel: 800-927-4669;
 Tel: +1 903-675-9321
 www.argonmedical.com

EMERGO EUROPE
 Prinsessegracht 20
 2514 AP The Hague
 The Netherlands
 +31 70 345 8570

The symbols glossary is located electronically at
www.argonmedical.com/symbols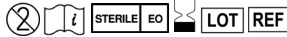

ENGLISH

INDICATIONS FOR USE:

The MaxiCELL™ Biopsy Needle is used to obtain cellular samples from soft tissue such as liver, kidney, prostate, breast, etc.

CONTRAINDICATIONS: Physician judgment is required when considering biopsy on patients with bleeding disorders, or receiving anti-coagulant medications who may have the potential of an increased risk of bleeding complications.

CAUTION:

- This device was designed, tested and manufactured for single use only. Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury. Do not reuse, reprocess or re-sterilize this device.
- For use only for fine needle biopsy of soft tissue as determined by a licensed physician.
- The instrument should be used by a physician familiar with the possible side effects, typical findings, limitations, indications and contraindications of a Fine Needle Aspiration procedure.
- After use, this product may be a potential biohazard. Handle in a manner which will prevent accidental puncture. Dispose in accordance with applicable laws and regulations.
- Do not use the device if package is open or damaged.
- Verify the integrity of the needle before inserting. If needle is damaged, replace the needle.

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

NOTE: These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

Store: in a cool, dry area.

Eng: MaxiCELL™ Biopsy Needle
 Ger: MaxiCELL™ Biopsienadel
 Fre: Aiguille de biopsie MaxiCELL™
 Spa: Aguja para biopsias MaxiCELL™
 Ita: Ago per biopsia MaxiCELL™
 Swe: MaxiCELL™ biopsinål
 Por: Agulha de Biopsia MaxiCELL™
 Gre: Βελόνα Βιοψίας MaxiCELL™
 Dut: MaxiCELL™ Biopsienaald
 Fin: MaxiCELL™-biopsianeula
 Nor: MaxiCELL™-biopsinål
 Dan: MaxiCELL™ biopsinål

BIOPSY PROCEDURE:

1. Using sterile technique, prepare the biopsy puncture site.
2. Insert the MaxiCELL™ needle through the skin with the stylet in place, and advance the needle, typically under imaging guidance, directly into the lesion with the scraping aperture at the 12 o'clock position. The scraping aperture is visible on direct inspection of the needle prior to insertion and there is a notch on the stylet which marks the position of the aperture.
3. Remove the stylet and attach a syringe.
4. While applying suction, gently **angle the needle shaft over forefinger to create a convex needle shape which opens the side port of the needle to engage the scraping aperture in the lesion** and move the needle up and down rapidly several times within the lesion.
5. When the aspiration procedure is complete, maintain gentle suction and **angle the MaxiCELL™ needle under the forefinger to create a concave shape relative to the scraping aperture and thus disengage the scraping aperture to allow for easy withdrawal.**
6. The sample (aspirate or tissue fragments) are now contained within the needle and ready to be deposited on a slide or in formalin for cytological analysis.
7. Remove the MaxiCELL™ Biopsy Needle.

BIOPSY PROCEDURE w/ INTRODUCER:

1. Using sterile technique, prepare the biopsy puncture site.
2. Using imaging guidance, insert the introducer through the skin with stylet in place and position proximal to the lesion.
3. Remove the stylet.
4. Insert the MaxiCELL™ needle through the introducer with the stylet in place, and advance the needle directly into the lesion with the scraping aperture at the 12 o'clock position. The scraping aperture is visible on direct inspection of the needle prior to insertion and there is a notch on the stylet which marks the position of the aperture.
5. Remove the stylet and attach a syringe.
6. While applying suction, move the needle forward and backward rapidly several times within the lesion.
7. When the aspiration procedure is complete, maintain gentle suction and remove the MaxiCELL™ Biopsy Needle.
8. The sample (aspirate or tissue fragments) are now contained within the needle and ready to be deposited on a slide or in formalin for cytological analysis.
9. Remove the introducer needle.

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