**ARGON MEDICAL DEVICES, INC.**

1445 Flat Creek Road  
Athens, Texas 75751 USA  
Tel: +1 (903) 675 9321  
Tel: +1 (800) 927 4669  
[www.argonmedical.com](http://www.argonmedical.com)

**EMERGO EUROPE**

Prinsessegracht 20  
2514 AP The Hague  
The Netherlands  
+31 70 345 8570

The symbols glossary is located electronically at  
[www.argonmedical.com/symbols](http://www.argonmedical.com/symbols)

en	Instruction For Use
bg	Инструкции за употреба
cs	Návod k použití
da	Brugsanvisninger
de	Gebrauchsanweisung
el	Οδηγίες χρήσης
es	Instrucciones de uso
et	Kasutusjuhised
fi	Käyttöohje
fr	Mode d'emploi
hu	Használati utasítás
it	Istruzioni per l'uso
lt	Naudojimo instrukcijos
lv	Lietošanas norādījumi
nl	Gebruiksaanwijzing
no	Bruksanvisning
pl	Instrukcja użycia
pt	Instruções de Utilização
ro	Instrucțiuni de utilizare
sk	Návod na použitie
sv	Bruksanvisning

**SEMI-AUTOMATED / SINGLE ACTION BIOPSY  
DEVICE (SABD™)**

*Bevel and Trocar Tip Design*

**Intended Use:**

To be used for soft tissue and tumor biopsy of such organs as the liver, spleen, kidney, prostate, lung, breast and lymph nodes. When used for breast biopsy, product is for diagnosis only.

**Contraindications:**

Sound professional judgment should be used when performing biopsies on patients who are using anticoagulant therapy and/or patients with known hemophilia problems.

**Cautions:**

- **Rx Only:** Federal Law (USA) restricts this device to sale by or on the order of a physician. Read instructions prior to use.
- The SABD™ device is designed for **single use only**.
- Do not re-use, re-process, or re-sterilize the tray or any items provided in the tray.
- Re-use, re-processing, or re-sterilization may compromise the structural integrity and the intended function of the product which could result in patient injury.
- Verify the integrity of all items before use. Do not use the contents if package is open or damaged.

**Warnings:**

- Device should only be used by or under supervision of a physician who is completely familiar with the use and limitations of biopsy sampling.
- These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.
- Functional tests are performed during manufacture. Do not "dry fire" device.
- Use sound professional judgment when deciding whether it is advisable to use the product with the aid of visualization modalities such as ultrasound, computerized tomography (CT), fluoroscopy or direct visualization.
- Not for use with MRI
- Ensure that no critical structures are beyond tip of device within needle stroke length (throw).

**Potential Complications:**

Possible complications may include, but are not limited to the following:


- Hemorrhage
- Infection
- Adjacent Tissue Injury
- Pain
- Hemoptysis
- Hemothorax

- Organ or Vessel Perforation
- Pneumothorax
- Air Embolism

### How Supplied:

The SABD™ device is supplied sterile by ethylene oxide gas as a single use product. The device is packaged in a plastic pouch. Do not use the device if package is open or appears to be damaged or defective. The device has no components made of natural rubber latex.

### Preparation and Instructions for Use:

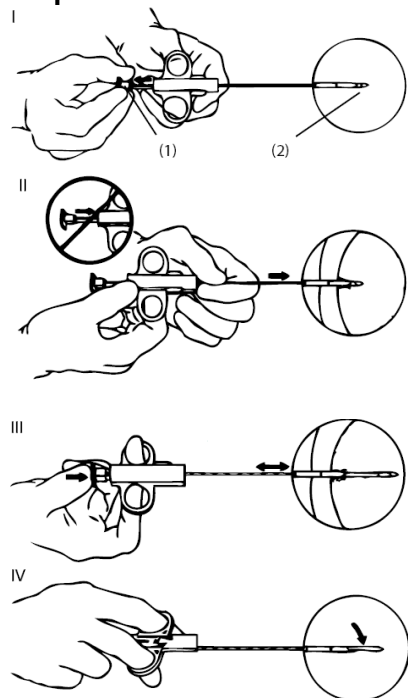- I. Ready the device by pulling back on the plunger (1) until a firm click is felt, indicating that the cutting cannula is locked into "Ready" position.
- II. With the plunger (1) fully retracted, so that specimen notch is completely covered by the cutting cannula, advance Device Tip (2) proximal to the area to be biopsied. **Note: Do not advance Plunger (1) until stylet is in position.**
- III. While maintaining the Device Tip (2) position, advance the Plunger (1) with thumb to expose specimen notch within area to be biopsied. Activate cutting cannula by fully depressing Plunger (1) with thumb to capture tissue within specimen notch.
- IV. Remove the device from the biopsy area. To retrieve tissue specimen, pull back Plunger (1) until a firm click is felt, indicating cutting cannula is locked into position. Advance Plunger (1) forward to expose tissue specimen within notch. Retrieve tissue from specimen notch.

### Handling and Disposal:

Handle in a manner which will prevent accidental puncture, percutaneous injuries and exposure to bloodborne pathogens. After use, this product may be a

potential biohazard. Dispose in accordance with applicable laws and regulations.

### Storage:

Store at standard ambient temperature.

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