
en	Instruction For Use
bg	Инструкции за употреба
cs	Pokyny pro použití
da	Brugsanvisning
de	Gebrauchsanweisung
el	Οδηγίες χρήσης
es	Instrucciones de uso
et	Kasutusjuhised
fi	Käyttöohjeet
fr	Mode d'emploi/Notice d'utilisation
hu	Használati utasítás
it	Istruzioni per l'uso
lt	Naudojimo instrukcijos
lv	Lietošanas instrukcija
nl	Gebbruiksaanwijzing
no	Bruksanvisning
pl	Instrukcja obsługi
pt	Instruções de utilização
ro	Instrucțiuni de utilizare
sk	Návod na použitie
sv	Bruksanvisning
tr	Kullanım talimatları

**ARGON MEDICAL DEVICES, INC.**

1445 Flat Creek Road  
Athens, Texas 75751 USA  
Tel: +1 800-927-4669  
Tel: +1 903-675-9321  
www.argonmedical.com

EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands  
+ 31 70 345 8570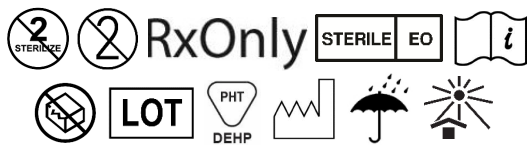

The symbols glossary is located electronically at [www.argonmedical.com/symbols](http://www.argonmedical.com/symbols)

IFU7573/1019C

**en - English****Indication for use**

The product is for drainage using Seldinger technique

**Duration**

Up to 12 weeks

**Warnings**

- This device was designed, tested and manufactured for single use only. Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury. Do not reuse, reprocess or re-sterilize this device.
- Do not use if package appear open or if the expiry date has been exceeded.
- This product contains the phthalate DEHP. When using the device, it should be taken into consideration that possible health affects when used for long durations in pregnant women, nursing mothers and children cannot be entirely discounted.
- The product must only be used by qualified personnel who are familiar with the technique.

**Preparation**

- Loosen the straightener together with the guidewire.
- Inject saline with syringe in a vertical position.
- Advance the straightener carefully over the curvature of the catheter, while straightening the curvature with your fingers.
- Gently pull the thread to lock the pigtail in order to avoid unintentional looping of the thread.
- Introduce the stiffener and fasten it onto the catheter (Luer Lock).
- Remove the straightener
- Activate the coating by saline.

**Procedure**

- Perform the skin incision under local anaesthesia.
- Introduce the needle into the cavity in question using ultrasound guidance, CT or fluoroscopy.
- Verify correct position by aspiration or by injection of contrast medium and fluoroscopy.
- When correctly positioned introduce the guidewire through the needle.
- Withdraw the needle over the guidewire.
- Dilate the canal.
- Advance the catheter over the guidewire in fluoroscopy.
- When the catheter tip is inside the cavity, loosen the stiffener.
- Hold the stiffener tight and advance the catheter over the stiffener until the complete length of the pigtail is in the cavity.
- Remove the stiffener and the guidewire.
- Gently pull the thread to ensure correct position and to complete curvature of the pigtail.
- Fix the thread tightly by winding it around the slot in the hub and finish by pressing the clip onto the slot and twist off the handle from the clip.
- Connect the catheter to a drainage bag using a suitable connecting tube.
- Verify correct position using fluoroscopy.

It is recommended affixing the catheter in a straight line and that any curvature be applied to the connecting tube.

It is recommended that the physician follow the hospital's standard of care procedures for drainage catheters.

**Removal of catheter**

- Disconnect the drainage bag connector tube from catheter.
- Remove the clip and unwind the suture. Check that both threads are loose and cut one thread in order to loosen the pigtail.
- Pull the catheter out gently. If access is to be maintained, a straight floppy tip guidewire passed through the catheter will facilitate removal while maintaining access.

**Caution**

If the catheter is to be removed in another department, we recommend that these guidance notes accompany the patient's case notes to ensure that the relevant personnel are aware of the presence of a locking catheter. It is also advisable to inform the patient.

**Storage**

Store in a dark, dry, cool place. Avoid extended exposure to light.

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