en	Instruction For Use
bg	Инструкции за употреба
cs	Pokyny pro použití
da	Brugsanvisning
de	Gebrauchsanweisung
el	Οδηγίες χρήσης
es	Instrucciones de uso
et	Kasutusjuhised
fi	Käyttöohjeet
fr	Mode d'emploi
hu	Használati utasítás
it	Istruzioni per l'uso
lt	Naudojimo instrukcijos
lv	Lietošanas instrukcija
nl	Gebbruiksaanwijzing
no	Bruksanvisning
pl	Instrukcja obsługi
pt	Instruções de utilização
ro	Instrucțiuni de utilizare
sk	Návod na použitie
sv	Bruksanvisning
tr	Kullanım talimatları

ARGON MEDICAL DEVICES, INC.
1445 Flat Creek Road
Athens, Texas 75751 USA
Tel: 800-927-4669
Tel: +1 903-675-9321
www.argonmedical.com

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands
+31 70 345 8570

The symbols glossary is located electronically at www.argonmedical.com/symbols

CE 2797

IFU7653/0319C

Indication for use

The product is for biliary drainage

Duration

Up to 12 weeks

Warnings

- This device was designed, tested and manufactured for single use only. Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury. Do not reuse, reprocess or re-sterilize this device.
- Do not use if package appear open or if the expiry date has been exceeded.
- The product must only be used by qualified personnel who are familiar with the technique.

Preparation

- Loosen the straightener together with the guidewire.
- Advance the straightener carefully over the curvature of the catheter, while straightening the curvature with your fingers.
- Gently pull the thread to lock the pigtail in order to avoid unintentional looping of the thread.
- Introduce the chosen stiffener (if plastic: activate the hydrophilic coating with saline) and fasten the stiffener onto the catheter (Luer Lock).
- Remove the straightener
- Activate the coating by saline.

Procedure

- Select and prepare the drainage site using standard technique.
- Perform the skin incision under local anaesthesia.
- Perform a fine needle puncture using ultrasound or X-ray.
- When correctly positioned remove the stylet and introduce the 0.018" guidewire.
- Remove the needle and dilate with the assembled dilator system.
- Loosen the inner catheter (coloured hub from white) checking that the tip of the outer dilator has been placed inside the cavity (note the radiopaque ring marker).
- Remove the inner dilator.
- Introduce a 0.035" guidewire.
- Remove the outer dilator and withdraw the thin guidewire.
- Dilate the tract, if necessary.
- Advance the catheter over the guidewire in fluoroscopy.
- When the catheter tip is inside the biliary tree, loosen the metal stiffener and replace it by the hydrophilic plastic stiffener. Advance the catheter over the guidewire as far as duodenum.
- The radiopaque marker indicates the last side hole and must be positioned in the biliary passage.
- Remove the stiffener and the guidewire.
- Gently pull the thread to ensure correct position and to complete curvature of the pigtail.
- Fix the thread tightly by winding it around the slot in the hub and finish by pressing the clip onto the slot and twist off the handle from the clip.
- Connect the catheter to a drainage bag using a suitable connecting tube.
- Verify correct position of the catheter using fluoroscopy.

It is recommended affixing the catheter in a straight line and that any curvature be applied to the connecting tube.

It is recommended that the physician follow the hospital's standard of care procedures for drainage catheters.

Removal of catheter

- Disconnect the drainage bag connector tube from catheter.

SKATER® Introducer Biliary Drainage Kit Locking

- Remove the clip and unwind the suture. Check that both threads are loose and cut one thread in order to loosen the pigtail.
- Pull the catheter out gently. If access is to be maintained, a straight floppy tip guidewire passed through the catheter will facilitate removal while maintaining access.

Caution

If the catheter is to be removed in another department, we recommend that these guidance notes accompany the patient's case notes to ensure that the relevant personnel are aware of the presence of a locking catheter. It is also advisable to inform the patient.

Storage

Store in a dark, dry, cool place. Avoid extended exposure to light.

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