ARGON MEDICAL DEVICES, INC.
1445 Flat Creek Road Athens, Texas 75751 USA
Tel: 800-927-4669;
Tel: +1 903-675-9321
www.argonmedical.com

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands
+31 70 345 8570

The symbols glossary is located electronically at
www.argonmedical.com/symbols

Eng: Suggested Procedure: Prostate Stabilization Needle Set
Ger: Vorschlag Zur Vorgehensweise: Nadelset Zur Prostatastabilisierung
Fre: Procédure Suggérée: Aiguille De Stabilisation De La Prostate
Spa: Procedimiento Sugerido: Estabilización Prostática Juego De Agujas
Ita: Procedimento Sugerito: Ago Per La Stabilizzazione Della Prostata
Swe: Rekommenderad Metod: Nålsats För Prostatastabilisering
Por: Procedimento Sugerido: Conjunto De Agulhas De Estabilização Da Próstata
Gre: Συνιστωμενη Διαδικασια: Σετ Βελόνας Σταθεροποίησης Του Προστάτη
Dut: Volg Deze Procedure: Prostaat Stabilisatie Naaldset
Fin: Suositeltu Käyttötapa: Neulasarja Eturauhasen Stabilointiin
Nor: Foreslått Fremgangsmåte: Nålesett For Prostatastabilisering.
Dan: Foreslået Procedure: Prostata Stabiliseringskanyle Sæt

Eng:

The barb is deployed out of the side eyelet which is located 2 cm proximal to the needle point and aligned with the point of the cap.

- Both needles are inserted into the prostate, under guidance, to the appropriate location.
- Each barb is deployed by holding the needle hub and pulling the stylet cap back approximately 1 cm (locking pin snaps up).
- Allow the prostate to settle into its bed.
- While holding the needle hub, gently squeeze the locking spring and slide it down the needle until it firmly contacts the grid. Repeat this procedure for both needles.
- To remove each needle from the prostate, hold the needle hub and push the stylet cap forward until it is fully against the cannula hub. The stylet cap may need to be rotated slightly while pushing forward to release the locking pin. Remove the needle.

Caution: For Single Patient Use Only. Do not attempt to clean or resterilize this product. After use, this product may be a potential biohazard. Handle in a manner which will prevent accidental puncture. Dispose in accordance with applicable laws and regulations.

Note: These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

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