ARGON MEDICAL DEVICES, INC.
1445 Flat Creek Road
Athens, Texas 75751 USA
Tel: +1 800-927-4669;
Tel: +1 903-675-9321
www.argonmedical.com

Eng: BONE MARROW BIOPSY/ASPIRATION NEEDLE
Spa: AGUJA PARA BIOPSIA Y ASPIRACIÓN DE MÉDULA ÓSEA
Fre: AIGUILLE D'ASPIRATION / BIOPSIE DE MOELLE OSSEUSE
Por: AGULHA PARA BIOPSIA E ASPIRAÇÃO DA MEDULA ÓSSEA
Ger: KNOCHENMARKBIOPSIE- UND ASPIRATIONSNADDEL
Dut: BIOPSIE-/ASPIRATIENAALD VOOR BEENMERG
Ita: AGO PER ASPIRAZIONE/BIOPSIA DEL MIDOLLO OSSEO
Swe: BIOPSI-/ASPIRATIONSNÄL FÖR BENMÄRGSPROV
Gre: ΒΕΛΟΝΑ ΒΙΟΨΙΑΣ/ΑΝΑΡΡΟΦΗΣΗΣ ΜΥΕΛΟΥ ΤΩΝ ΟΣΤΩΝ
Fin: BIOPSIA/ASPIRAATIONEULA LUUYDINNÄYTTEIDEN OTTAMISEEN
Dan: BIOPSI/ASPIRATIONSKANYLE TIL KNOGLEMARV/SBIOPSI
Nor: BIOPSI/ASPIRASJONSNAL FOR BENMARGSPRØVE
Tur: KEMİK İLİĞİ BİYOPSİ/ASPIRASYON İGNEİ

EC REP

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands
+31 70 345 8570

STERILE EO

ENGLISH

Indications for Use: Intended for the purpose of harvesting bone and/or bone marrow specimens.

Contraindications: For use only for biopsies of bone/bone marrow as determined by a licensed physician. These needles should be used by a physician familiar with the possible side effects, typical findings, limitations, indications and contraindications of bone biopsy. Physician judgment is required when considering biopsy on patients with bleeding disorder, or receiving anti-coagulant medications.

Caution: For Single Patient Use Only. Do not attempt to clean or resterilize this product. After use, this product may be a potential biohazard. Handle in a manner which will prevent accidental puncture. Dispose in accordance with applicable laws and regulations.

Note: These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

Aspiration Procedure:

- Place the patient in a right or left lateral position, with the back comfortably flexed and the top knee drawn toward the chest.
- Locate the posterior superior iliac spine and mark.
- Using sterile technique, prepare the skin with antiseptic and drape.
- Infiltrate the marked area with local anesthetic, especially the periosteum.
- Make a skin incision with a scalpel blade over the marked area.
- Hold the needle with the proximal end in palm and the index finger against the shaft near the tip. This position stabilizes the needle and allows better control.
- Introduce the needle through the incision pointing toward the anterior superior iliac spine and bring it into contact with the posterior iliac spine.
- Using gentle, but firm pressure, advance the needle. Rotate the needle in an alternating clockwise-counterclockwise motion. Entrance into the marrow cavity is generally detected by decreased resistance.
- Remove stylet by rotating upper section of handle 90°, and pulling straight out.
- Attach a syringe with a luer taper to the hub of the Bone Marrow Biopsy/Aspiration Needle with a firm, push, twist motion.
- Apply suction by withdrawing syringe plunger. Remove the aspirated specimen collected from the syringe.

Biopsy Procedure:

Take the biopsy after the aspiration using the same skin incision, but choosing a different location to obtain the biopsy sample.

Repeat Steps 1 through 9 of the procedure and continue as follows:

- Optional - If desired, cannula can be capped with luer cap provided.
- Slowly and gently advance the needle a millimeter at a time, with clockwise-counterclockwise motion (for better cutting) until adequate marrow is obtained.
- Pull the needle back a few millimeters and, with less pressure, direct its tip at a slightly different angle. (Caution) Excessive pressure may cause needle to bend. Then advance the needle a few millimeters further. This procedure ensures that the specimen is severed before withdrawing the needle.
- Rotate the needle along its axis with quick, full twists-several times to the right and to the left. Slowly remove the needle with alternating rotary motions.
- For safety and convenience, Manan has introduced the Anti-Stick Probe Guide as a standard feature of the "J" type Bone Marrow Needle. The purpose of this device is two-fold: provide an easier method to align the probe in the needle tip for sample expulsion and protect the sharp tip to eliminate any accidental injury during sample expulsion. After sample is obtained, slip larger opening end of guide over the needle tip. Push guide down on needle until it fits snugly. Insert probe into smaller opening and push gently to expel sample from proximal end of needle. Remove probe from guide. If another sample is to be taken, remove guide and reassemble needle stylet and cannula. Wipe cannula and stylet tip clean. If no other samples are to be obtained, leave guide in place to cover tip of needle and discard needle assembly.

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