
ARGON MEDICAL DEVICES, INC.
1445 Flat Creek Road Athens, Texas 75751 USA
Tel: 800-927-4669; Tel: +1 903-675-9321
www.argonmedical.com


EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands

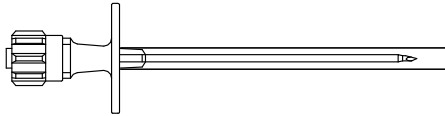Eng: Pediatric bone marrow access needles
Ger: Pädiatrische Knochenmarkzugriffsnadel
Fre: Aiguilles pédiatriques de moelle osseuse
Spa: Aguja pediátrica para el acceso de médula ósea
Ita: Aghi pediatrici per il prelievo del midollo osseo
Swe: Pediatrik åtkomstnål för benmärg
Por: Agulhas pediátricas para o acesso à medula óssea
Gre: Παιδιατρικές βελόνες πρόσβασης μυελου των οστών
Dut: Pediatrische toegangsnalden voor beenmerg
Fin: Pediatriiset luuydinneulat
Nor: Pediatrik tilgangsnål for benmarv
Dan: Pædiatrik kanylte for adgang til knoglemarv


1

Eng: Epiphyseal Plate
Ger: Epiphysäre
Wachstumszone
Fre: Cartilage De Conjugaison
Spa: Placa Epifisaria
Ita: Cartilagine Di
Coniugazione
Swe: Cartilago Epiphysialis
Por: Cartilagem Epifisária
Gre: Επιφυσική Πλάκα
Dut: Epifysaire Plaat
Fin: Cartilago Epiphysialis
Nor: Epifyseskiven
Dan: Epifyse Skivev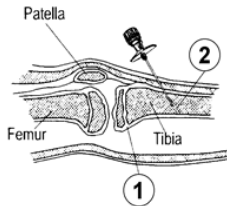


2

Eng: Tibial Tuberosity
Ger: Tibiale Tuberositas
Fre: Tubérosité Tibiale
Spa: Tuberosidad Tibia
Ita: Tuberosità Tibiale
Swe: Tuberositas Tibiae
Por: Tuberosidade Tibial
Gre: Κνημιαίο Κύρτωμα
Dut: Tuberositas Tibialis
Fin: Tuberositas Tibiae
Nor: Tuberositas Tibiae
Dan: Tuberositas Tibia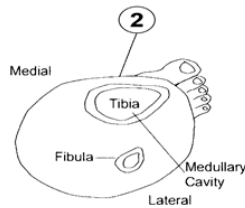

3

Eng: Medial Malleolus
Ger: Mediale Malleolus
Fre: Malleôle Interne
Spa: Maléolo Medial
Ita: Malleolo Mediale
Swe: Mediale Malleolen
Por: Maléolo Medial
Gre: Μεσαία Σφύρα
Dut: Mediale Malleolus
Fin: Mediale Malleolus
Nor: Malleolus Medialis
Dan: Medial Malleol

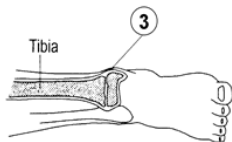Eng:

Indications for Use: For obtaining access into medullary cavities for the purposes of initiating resuscitative infusion or for aspirating marrow in pediatric patients.

Contraindications: For use only for biopsies of bone/bone marrow as determined by a licensed physician. These needles should be used by a physician familiar with the possible side effects, typical findings, limitations, indications and contraindications of bone biopsy. Physician judgment is required when considering biopsy on patients with bleeding disorder, or receiving anti-coagulant medications. This device should not be

used in patients who have a compromised extremity, skin infection or comminuted fracture at insertion site. This device also should not be used in those patients who have osteogenesis imperfecta or osteopetrosis.

Caution: For Single Patient Use Only. Do not attempt to clean or resterilize this product. After use, this product may be a potential biohazard. Handle in a manner which will prevent accidental puncture. Dispose in accordance with applicable laws and regulations.

Note: These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

Recommended Insertion Sites:

The preferred site is in the tibial tuberosity of the proximal tibia. The actual site is located 2cm below (inferior) the tuberosity.

A secondary site is the medial malleolus of the distal tibia.

Suggested Instructions for Insertion (for Proximal Tibial Tuberosity):

1. Stabilize limb in a semi-externally rotated position.
2. Prep area with iodine and/or alcohol.
3. Hold needle in place at right angle to the plate of the bone. The needle should be pointing anteroposterior with the leg semi-externally rotated. Ensure the needle is angled away from the knee joint. Special care should be taken to avoid the epiphyseal plate.
4. The insertion is done in two movements:
 - a) The first includes penetration of the skin and subcutaneous tissues.
 - b) The second includes penetration of the periosteum and the bone itself.
5. Using a firm twisting motion, pass the needle through the skin and cortex of the bone until an obvious "pop" or give is felt. There is usually no doubt as to the exact sensation.
6. Withdraw the stylet. It is common for a small amount of marrow and/or blood to seep out of the top of the needle after withdrawing the stylet.
7. Proper placement is confirmed by loss of resistance.
8. The needle will feel firmly fixed by the bone.
9. Aspiration should occur without resistance. It is important to stress that no material should be injected or infused until a "blood-marrow" mixture has been easily aspirated.
10. If necessary to further confirm location, inject 5cc of sterile saline and withdraw blood samples. Proper position can be assured by the free flow of blood during aspiration. Aspirate no more than 1cc of marrow.
11. Stabilize needle in a standard fashion at skin level and initiate infusion.

In the Event of Obstruction or Failure:

- a. Reassess insertion site.
- b. Flush cannula, re-insert stylet and repeat above procedure.
- c. If it is not possible to obtain aspirate or begin infusion, consider an alternate insertion site.

Needle Removal:

To remove the needle, apply gentle traction and rotate the needle cannula back and forth to disengage from bone cortex. Dress insertion site using accepted practices.

Special Considerations:

Most complications from intraosseous infusions occur after 24 hours of infusion. Since this procedure is intended as a "last-resort", it should only be used until conventional venous infusion can be initiated. Its use for longer than 24 hours is not recommended.

References:

J. Friery, K. Weiner: "Start an IV in that Bone," *Emergency*, November (1987), 28-33.

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