ARGON MEDICAL DEVICES, INC.
1445 Flat Creek Road
Athens, Texas 75751 USA
Tel: +1 800-927-4669;
Tel: +1 903-675-9321
www.argonmedical.com

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands
+31 70 345 8570

The symbols glossary is located electronically at
www.argonmedical.com/symbols

ENGLISH

Indications for Use: For Sternal or Iliac Crest Aspiration.

Contraindications: For use only for biopsies of bone/bone marrow as determined by a licensed physician. These needles should be used by a physician familiar with the possible side effects, typical findings, limitations, indications and contraindications of bone biopsy. Physician judgment is required when considering biopsy on patients with bleeding disorder, or receiving anti-coagulant medications.

Caution: For Single Patient Use Only. Do not attempt to clean or resterilize this product. After use, this product may be a potential biohazard. Handle in a manner which will prevent accidental puncture. Dispose in accordance with applicable laws and regulations.

Note: These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

Instructions for Use:

- Using sterile technique, prepare the skin with antiseptic and drape.
- Infiltrate the selected area with local anesthetic.

For Sternal Aspiration (Steps 3-4):

- Adjust the needle stop for correct needle length. The adjustable screw-stop allows for a minimum needle length of 3/8"; when the screw-stop is removed, the needle length is 1-7/8".

Note: Do not use for sternal puncture with stop assembly removed. (See Iliac Crest instructions).

- Insert needle into the sternum in usual fashion. Proceed with Step 7.

For Iliac Crest Aspiration (Steps 5-6):

- The screw-stop can be removed from the screw thread for a needle length of 1-7/8". If more length is desired, screw stop all the way up (with needle guard in place), grab stop assembly and pull briskly straight along needle. The entire needle stop assembly will now be removed and the needle length will be 2-5/8".

Note: Do not use if needle is bent.

- Insert needle into Iliac crest in usual fashion.
- Once needle tip is positioned in marrow cavity, remove stylet by rotating upper section of handle 90°, and pulling straight out.
- A luer-lock connection is provided for syringe connection.
Note: Either luer-lock or luer-slip syringe may be used.
- Aspirate marrow and process samples in accordance with hospital procedure.
- Dispose of all used needles in a correct and safe manner.

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