ARGON MEDICAL DEVICES, INC.
1445 Flat Creek Road
Athens, Texas 75751 USA
Tel: 800-927-4669;
Tel: +1 903-675-9321
www.argonmedical.com

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands
+31 70 345 8570

The symbols glossary is located electronically at
www.argonmedical.com/symbols

ENGLISH

Indications for Use: Galactogram Kit is intended for the purpose of performing Breast Galactography.

Contraindications: For use only for Breast Galactography as determined by a licensed physician. This kit should be used by a physician familiar with the possible side effects, typical findings, limitations, indications and contraindications of Breast Galactography.

Caution: For Single Patient Use Only. Do not attempt to clean or resterilize this product. After use, this product may be a potential biohazard. Handle in a manner which will prevent accidental puncture. Dispose in accordance with applicable laws and regulations.

This device was designed, tested and manufactured for single patient use only. Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness and/or injury. DO NOT reuse, reprocess or re-sterilize this device.

Note: These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

Suggested Galactography Procedure:

- Identify the duct
Before attempting to perform galactography, first make sure that a discharge can be identified from a single opening on the nipple. Clean the nipple with alcohol. Use of an incandescent lamp to warm the nipple will help to relax the sphincter, thus making cannulation of the duct easier. Use of a dissecting microscope, or a head mounted magnifying lens is helpful.
- Dilate the duct
After the nipple has had a chance to relax, manipulate out a very small amount of the discharge to identify the exact orifice to be cannulated. Use a small diameter (000000/.010" or 00000/.012") dilator. Gently rotate the dilator between the index finger and thumb as you insert it into the opening. Dilate with increasingly larger dilators. Once the abnormal duct is dilated, the discharge will usually flow more freely.

RTD001X				DGK002X, DGK002SX			
Color	Size	Color	Size	Color	Size	Color	Size
Blue	- 000000	Turquoise	- 0000	Green	- 00	Blue	- .010"
White	- 00000	Black	- 000	Yellow	- 0	White	- .012"

- Cannulate the duct
Fill the 1cc tuberculin syringe with water soluble contrast (undiluted Conray 60). Attach the blunt tipped cannula to the tuberculin syringe. Remove any air bubbles, as these will appear as filling defects on the galactogram. Gently place the cannula into the duct no deeper than 0.5 to 1cm. Gently inject 0.2 to 0.3cc contrast, or until there is slight resistance or some discomfort. Remove the cannula and obtain mammograms in 2 views with minimal compression. If the duct is not completely filled, or if the diagnosis is not clear, repeat the injection and mammograms. Magnification views can also be helpful.

If extravasation has occurred, the procedure should be discontinued and rescheduled in two to three weeks.

References

Tabar, L., Dean, P. *Interventional radiologic procedures in the investigation of lesions of the breast. Radiol Clin North Am* 1979; 17:607-621.
Tabar, L., Dean, P., Pentek, Z. *Galactography: the diagnostic procedure of choice for nipple discharge. Radiology, 1983; 149:31-38.*

Eng:	Galactogram Kit
Ger:	GalktogrammSet
Fre:	Trousse De Galactographie
Spa:	Equipo Para Galactogramas
Ita:	Kit Per Galattografia
Swe:	Galaktogramutrustning
Por:	Kit Para Galactografia
Gre:	Σετ Γαλακτογραφήματος
Dut:	Galactogram Kit
Fin:	Galaktogrammilaite
Nor:	Galaktogramsett
Dan:	Galactografi Sæt
Tur :	Galaktografi Seti
Rus :	Набор для галактограммы

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