ARGON MEDICAL DEVICES, INC.
 1445 Flat Creek Road
 Athens, Texas 75751 USA
 Tel: 800-927-4669;
 Tel: +1 903-675-9321
 www.argonmedical.com

ENG :

Intended Use: Intended for percutaneous nephrostomy drainage.

Contraindications: Only qualified personnel who are familiar with the technique must use the product. **Please see Removal of Catheter.**

FRENCH	MAXIMUM GUIDEWIRE SIZE	CATHETER OD	CATHETER STIFFENER
6	0.035"	2.14 mm	METAL
8	0.038"	2.80 mm	METAL AND PLASTIC
10	0.038"	3.47 mm	METAL AND PLASTIC
12	0.038"	4.12 mm	METAL AND PLASTIC
14	0.038"	4.78 mm	METAL AND PLASTIC

Preparation:

1. Straighten the pigtail out with the pigtail straightener. Gently pull the suture to avoid unintentional looping of the suture.
2. Introduce the metal or the plastic stiffening cannula (physician preference). Lock the metal stiffening cannula onto the catheter (Luer Lock). Remove the pigtail straightener.
3. To activate the hydrophilic coating, wet the catheter in sterile water or saline. Keep the catheter wet during placement. Use a wet gauze pad to handle the catheter during placement if necessary. Do not wipe the catheter with dry gauze or any solvents as it may damage the coating.

Procedure:

1. Select and prepare the drainage site using conventional technique.
2. Using local anesthesia and Seldinger Technique, insert a guidewire within the collecting system.
3. Progressively dilate the tract to 1 French size greater than the size of the catheter. This will ease introduction.
4. Wet the hydrophilic coated catheter to activate the coating (see "Preparation").
5. Using fluoroscopy, the catheter and metal stiffening cannula are advanced over the guidewire into the superficial part of the dilated pelvis. At the point of entry into the renal pelvis, the metal stiffening cannula is unlocked and held stationary while the catheter is further advanced into the pelvis over the guidewire. The catheter is advanced so that all the drainage holes are within the collecting system.
6. The metal stiffening cannula is removed.
7. Under fluoroscopy, slowly remove the guidewire while rotating the catheter counterclockwise. This movement will ensure correct position and cause the catheter to re-form the pigtail.
8. To lock the pigtail in its position:
 - Pull the thread gently.
 - Wind the thread around the slot and press the clip onto the slot. The thread must be stretched during the process.
9. Connect the catheter to a drainage bag with connector tubing (Argon Medical Devices Catalog # DBAG600).
10. Check the position of the catheter using fluoroscopy.

Removal of Catheter:

1. Loosen the clip, unwind the thread, and pull both threads in order to loosen the threads from the valve.
2. Cut one of the threads close to the hub. When the thread disappears the pigtail is loose. If you feel any kind of resistance do contact your X-ray department.
3. Pull the catheter out gently. If access is to be maintained, a straight floppy tip guidewire passed through the catheter will facilitate removal while maintaining access.

Storage: Store in a cool, dry area.

ENG: Skater® Locking Nephrostomy Drainage Catheter

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