**ARGON MEDICAL DEVICES, INC.**  
 1445 Flat Creek Road, Athens, Texas 75751 USA  
 Tel: 800-927-4669;  
 Tel: +1 903-675-9321  
 www.argonmedical.com

**STERILE EO**

**ENG:**

**Indications for Use:** HSG Catheters are intended for use in the injection of contrast material in the examination of the uterus and fallopian tubes.

**Contraindications:** Physician judgment should be used in determining in which patients Hystero-Salpingography or Hystero-Sonography procedures are contraindicated. These procedures are not recommended for patients with active pelvic infections.

**Potential Complications include:**

- Bleeding

**Warnings:**

1. Do not use Lipiodol or other oil based contrast media.
2. Do not use this device with a power injector for the delivery of the contrast media.
3. These devices should be used only by a physician familiar with the possible side effects, typical findings, limitations, indications and contraindications of Hystero-Salpingography and Hystero-Sonography.
4. **For Single Patient Use Only.** Do not attempt to clean or resterilize this product. After use, this product may be a potential biohazard. Dispose in accordance with applicable laws and regulations.

**Preparations:**

1. Refer to the following chart to aid in selecting the proper catheter:

Catalog No.	Accepts GW	Balloon Dia. W/ air	Balloon Dia. W/ saline
660005040 (5.5Fr)	0.035" (0.89mm)	11mm with 2.0ml air	10mm with 0.8ml saline
660007040 (7Fr)	0.038" (0.96mm)	13mm with 3.0ml air	12mm with 1.2ml saline

2. Check to insure that both the blue stopcock (balloon port) and the white stopcock (contrast media port) operate freely. Set both stopcocks to the open position.
3. Inflate and deflate the balloon (extend and retract the plunger on the vented syringe) to insure its functionality and integrity.

**NOTE: When inflating balloon with saline solution (instead of air), do not exceed the volumes specified in the above table.**

4. Expel air from the catheter by filling it with contrast media through the white stopcock. Close the white stopcock.
5. Remove the contrast media syringe to facilitate easier procedural handling.

**Instructions:**

1. Advance the introduction tube forward until the distal end of the catheter protrudes slightly from the distal end of the sheath.
2. Visualize the external cervical Os with the aid of a speculum and advance the introduction tube/catheter such that the tip of the catheter enters the cervical canal.
3. Holding the introduction tube stationary, advance the catheter into the cervical canal 1-2 cm.
4. **Slowly** inflate the balloon, providing occlusion, without causing patient discomfort.
5. If uterine placement is desired, advance the catheter into the uterus, inflate the balloon and gently withdraw against the internal Os.
6. Attach the contrast media syringe, open the white stopcock, inject the contrast liquid and conduct the examination.
7. When the examination has been completed, deflate the balloon by opening the blue stopcock and retracting the syringe plunger.
8. Remove the catheter and introduction tube.

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