ARGON MEDICAL DEVICES, INC.
1445 Flat Creek Road, Athens, Texas 75751 USA
Tel: 800-927-4669;
Tel: +1 903-675-9321
www.argonmedical.com

ENGLISH

Intended Use: Indicated for percutaneous biliary drainage.

Contraindications: Only qualified personnel who are familiar with the technique should use the product. **Please see Removal of Catheter.**

FRENCH	MAXIMUM GUIDEWIRE SIZE	CATHETER OD	CATHETER STIFFENER
8	0.038"	2.80 mm	METAL AND PLASTIC
10	0.038"	3.47 mm	METAL AND PLASTIC
12*	0.038"	4.12 mm	METAL AND PLASTIC

*available with locking pigtail or non-locking pigtail

Duration: For up to 12 weeks.

Warnings:

- Reuse or reprocessing of this instrument may lead to its failure and subsequent patient injury. Reprocessing and/or re-sterilization of this instrument may create the risk of contamination and patient infection. **DO NOT** reuse, reprocess or re-sterilize this device.
- If the catheter becomes malpositioned or if drainage ceases, the catheter should be promptly exchanged or removed.

Precautions:

- Physicians should use caution with patients who have bleeding diathesis, uncontrolled hypertension or who use anticoagulants.
- Patients with indwelling drainage catheters should be evaluated routinely to ensure continuous function of the catheter.
- Product is intended for use by qualified personnel who are familiar with its use and techniques for insertion and removal.

Preparation:

- Straighten the pigtail out with the pigtail straightener. Gently pull the suture to avoid unintentional looping of the suture.
- Introduce the metal stiffening cannula (if plastic stiffening cannula is used, activate the hydrophilic coating with saline). Lock the metal stiffening cannula onto the catheter (Luer Lock). Remove the pigtail straightener.
- To activate the hydrophilic coating, wet the catheter with sterile water or saline. Keep the catheter wet during placement. Use a wet gauze pad to handle the catheter during placement if necessary. Do not wipe the catheter with dry gauze or any solvents as it may damage the coating.

Procedure:

- Select and prepare the drainage site using conventional technique.
- Using local anesthesia and the Seldinger Technique, insert a guidewire into the Biliary Tree.
- Progressively dilate the tract to 1 French size greater than the size of the catheter. This will ease introduction.
- Wet the hydrophilic coated catheter to activate the coating (see "Preparation").
- Under fluoroscopy, the catheter and metal stiffening cannula are advanced over the guidewire into the Biliary Tree. At the point of entry into the Biliary Tree, the metal stiffening cannula is unlocked, removed and replaced with the plastic stiffening cannula. The catheter is advanced over the guidewire as far as the duodenum.
- Remove the plastic stiffening cannula.
- Under fluoroscopy, slowly remove the guidewire while rotating the catheter counterclockwise. This movement will ensure correct position and cause the catheter to reform the pigtail.
- If using the Skater™ Biliary Drainage Catheter with Locking Pigtail, in order to lock the pigtail in its position:
 - Pull the thread gently.
 - Wind the thread around the slot and press the clip onto the slot. The thread must be stretched tight during the process.
- Connect the catheter to a drainage bag with connector tubing (Catalog # DBAG600).
- Check the position of the catheter using fluoroscopy.

Removal of Catheter:

- Disconnect the drainage bag connector tube from catheter.
- If using the Skater™ Biliary Drainage Catheter with Locking Pigtail, in order to unlock and loosen the pigtail:
 - Remove the clip from the hub and unwind the thread.
 - Check that both threads are loose.
 - Cut one thread in order to loosen the pigtail.
- Pull the catheter out gently. If access is to be maintained, a straight floppy tip guidewire passed through the catheter will facilitate removal while maintaining access.

Disposal: After use, this product may be a potential biohazard. Dispose in accordance with applicable laws and regulations.

Storage: Store in a cool, dry area.

ENG: Skater™ Biliary Drainage Catheter With Radiopaque Band

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