SKATER® Single Step Drainage Set - Locking

ARGON MEDICAL DEVICES, INC.

1445 Flat Creek Road
Athens, Texas 75751 USA
Tel: 800-927-4669
Tel: +1 903-675-9321
www.argonmedical.com

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands
+31 70 345 8570

The symbols glossary is located electronically at
www.argonmedical.com/symbols

en	Instruction For Use
bg	Инструкции за употреба
cs	Pokyny pro použití
da	Brugsanvisning
de	Gebrauchsanweisung
el	Οδηγίες χρήσης
es	Instrucciones de uso
et	Kasutusjuhised
fi	Käyttöohjeet
fr	Mode d'emploi
hu	Használati utasítás
it	Istruzioni per l'uso
lt	Naudojimo instrukcijos
lv	Lietošanas instrukcija
nl	Gebruiksaanwijzing
no	Bruksanvisning
pl	Instrukcja obsługi
pt	Instruções de utilização
ro	Instructiuni de utilizare
sk	Návod na použitie
sv	Bruksanvisning
tr	Kullanım talimatları

IFU7565/0319D

en - English**Indication for use**

The product is for single step drainage of cysts, gall bladders and nephrostomies.

Duration

Up to 12 weeks

Warnings

- This device was designed, tested and manufactured for single use only. Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury. Do not reuse, reprocess or re-sterilize this device.
- Do not use if package appear open or if the expiry date has been exceeded.
- During insertion avoid contact with bone, cartilage and scar tissue which can damage the catheter tip.
- The product must only be used by qualified personnel who are familiar with the technique.

Preparation

- Advance the straightener carefully over the curvature of the catheter, while straightening the curvature with your fingers.
- Gently pull the thread to lock the pigtail in order to avoid unintentional looping of the thread.
- Introduce the metal stiffener fully with the trocar needle slightly drawn back to avoid damaging the catheter wall.
- Fasten the stiffener onto the catheter and introduce the trocar needle. Support the tip of the catheter advancing the needle. Snap the Choice Lock® onto the stiffener hub.
- Remove the straightener and the catheter is now ready for use.
- Activate the coating by saline.

Procedure**Single-step catheterisation through a skin incision.**

- Perform the skin incision under local anaesthesia.
- Introduce the catheter set to the cavity using ultrasound guidance, CT or fluoroscopy.
- Verify correct position by releasing the Choice Lock® with one hand and remove the stylet. Check that fluid is escaping or aspirate with a 10 ml syringe. The stylet must always be correctly positioned if you want to change the placement. When correctly positioned advance the catheter over the needle, holding the needle steady.
- Ensure that the complete length of the pigtail is in the cavity.
- Gently pull the thread to ensure correct position and to complete curvature of the pigtail.
- Fix the thread tightly by winding it around the slot in the hub and finish by pressing the clip onto the slot and twist off the handle from the clip.
- Connect the catheter to a drainage bag using a suitable connecting tube.

It is recommended affixing the catheter in a straight line and that any curvature be applied to the connecting tube.

It is recommended that the physician follow the hospital's standard of care procedures for drainage catheters.

Removal of catheter

- Disconnect the drainage bag connector tube from catheter.
- Remove the clip and unwind the suture. Check that both threads are loose and cut one thread in order to loosen the pigtail.
- Pull the catheter out gently. If access is to be maintained, a straight floppy tip guidewire passed through the catheter will facilitate removal while maintaining access.

Caution

If the catheter is to be removed in another department, we recommend that these guidance notes accompany the patient's case notes to ensure that the relevant personnel are aware of the presence of a locking catheter. It is also advisable to inform the patient.

Storage

Store in a dark, dry, cool place. Avoid extended exposure to light.

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