en	Instruction For Use	دواعي الاستعمال
ar		
bg	Инструкции за употреба	
cs	Pokyny pro použití	
da	Brugsanvisning	
de	Gebrauchsanweisung	
el	Οδηγίες χρήσης	
es	Instrucciones de uso	
et	Kasutusjuhised	
fi	Käyttöohjeet	
fr	Mode d'emploi	
hu	Használati utasítás	
it	Istruzioni per l'uso	
lt	Naudojimo instrukcijos	
lv	Lietošanas instrukcija	
nl	Gebruiksaanwijzing	
no	Bruksanvisning	
pl	Instrukcja obsługi	
pt	Instruções de utilização	
pt-br	Instruções de uso	
ro	Instrucțiuni de utilizare	
sk	Návod na použitie	
sv	Bruksanvisning	
tr	Kullanım talimatları	

en - English

Indication for use

The product is for biliary drainage.

Duration

Up to 8 weeks

Warnings

- This device was designed, tested and manufactured for single use only. Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury. Do not reuse, reprocess or re-sterilize this device.
- Do not use if package appear open or if the expiry date has been exceeded.
- The product must only be used by qualified personnel who are familiar with the technique.

Preparation

Only for catheter with pigtail or ring configuration

- Advance the straightener carefully over the curvature of the catheter, while straightening the curvature with your fingers.
- When fully straighten ensure that the guidewire can pass through the catheter before use.

Procedure

1. Advance the catheter over the guidewire that has been introduced into the patient using standard technique.
2. When the catheter is fully loaded onto the guidewire remove the straightener.
3. Verify correct position of the catheter using fluoroscopy.
4. Remove the guidewire
5. Connect the catheter to a drainage bag using a suitable connecting tube.

We recommend bandaging the catheter in a straight line as it exits the skin. Any curvature required can be applied to the connecting tube.

Storage

Store in a dark, dry, cool place. Avoid extended exposure to light.


**ARGON MEDICAL DEVICES, INC.**

1445 Flat Creek Road
Athens, Texas 75751 USA
Tel: +1 (903) 675 9321
Tel: +1 (800) 927 4669
www.argonmedical.com

EC REP

EMERGO EUROPE

Prinsessegracht 20
2514 AP The Hague
The Netherlands
+31 70 345 8570

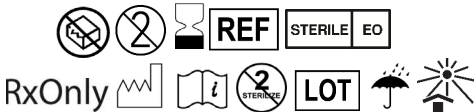The symbols glossary is located electronically at
www.argonmedical.com/symbols

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