en	Instruction For Use
bg	Инструкции за употреба
cs	Pokyny pro použití
da	Brugsanvisning
de	Gebrauchsanweisung
el	Οδηγίες χρήσης
es	Instrucciones de uso
et	Kasutusjuhised
fi	Käyttöohjeet
fr	Mode d'emploi
hu	Használati utasítás
it	Istruzioni per l'uso
lt	Naudojimo instrukcijos
lv	Lietošanas instrukcija
nl	Gebbruiksaanwijzing
no	Bruksanvisning
pl	Instrukcja obsługi
pt	Instruções de utilização
ro	Instrucțiuni de utilizare
sk	Návod na použitie
sv	Bruksanvisning
tr	Kullanım talimatları

**ARGON MEDICAL DEVICES, INC.**

1445 Flat Creek Road  
Athens, Texas 75751 USA  
Tel: +1 (903) 675 9321  
Tel: +1 (800) 927 4669  
[www.argonmedical.com](http://www.argonmedical.com)

**EMERGO EUROPE**

Prinsessegracht 20  
2514 AP The Hague  
The Netherlands  
+31 70 345 8570

The symbols glossary is located electronically at [www.argonmedical.com/symbols](http://www.argonmedical.com/symbols)

**CE** 2797

IFU7165/1019D

**Indication for use**

The product is for single step drainage of cysts, gall bladders and nephrostomies.

**Duration**

Up to 8 weeks

**Warnings**

- This device was designed, tested and manufactured for single patient use only. Reuse or reprocessing of this device may lead to its failure and subsequent patient injury. Reprocessing and/or resterilization of this device may create the risk of contamination and patient infection. Do not reuse, reprocess or resterilize this device.
- Do not use if package appears open or if the expiry date has been exceeded.
- During insertion, avoid contact with bone, cartilage and scar tissue which can damage the catheter tip.
- The product must only be used by qualified personnel who are familiar with the technique.

**Preparation**

- Advance the straightener carefully over the curvature of the catheter, while straightening the curvature with your fingers.
- Gently pull the thread to lock the pigtail in order to avoid unintentional looping of the thread.
- Introduce the metal stiffener fully with the trocar needle slightly drawn back to avoid damaging the catheter wall.
- Fasten the stiffener onto the catheter and introduce the trocar needle. Support the tip of the catheter by advancing the needle. Snap the Choice Lock™ onto the stiffener hub.
- Remove the straightener, and the catheter is now ready for use.

**Procedure****Single-step catheterization through a skin incision.**

- Perform the skin incision under local anaesthesia.
- Introduce the catheter set to the cavity using ultrasound guidance, CT or fluoroscopy.
- Verify correct position by releasing the Choice Lock™ with one hand and removing the stylet. Check that fluid is escaping or aspirate with a 10 ml syringe. The stylet must always be correctly positioned if you want to change the placement. When correctly positioned advance the catheter over the needle, holding the needle steady.
- Ensure that the complete length of the pigtail is in the cavity.
- Loosen the adapter and gently pull the thread turning the catheter clockwise to ensure correct position and to complete curvature.
- Fix the thread tightly by winding it around the slot in the adapter and finish up with pressing the clip onto the slot and twist off the handle from the clip.
- Verify correct position of the catheter using fluoroscopy.
- Remove the guidewire.

- Connect the catheter to a drainage bag using a suitable connecting tube.

It is recommended affixing the catheter in a straight line and that any curvature be applied to the connecting tube.

It is recommended that the physician follow the hospital's standard of care procedures for drainage catheters.

#### **Removal of catheter**

- Disconnect the drainage bag connector tube from catheter.
- Loosen the female/male adapter from the standard luer hub at the catheter to deactivate the valve.
- Remove the clip and unwind the suture. Check that both threads are loose and cut one thread in order to loosen the pigtail.
- Pull the catheter out gently. If access is to be maintained, a straight floppy tip guidewire passed through the catheter will facilitate removal while maintaining access.

#### **Caution**

If the catheter is to be removed on another ward, we recommend that these guidance notes accompany the patient's case notes to ensure that the relevant personnel are aware of the presence of a locking catheter. It is also advisable to inform the patient.

#### **Storage**

Store in a dark, dry, cool place. Avoid extended exposure to light.

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