en	Instruction For Use
bg	Инструкции за употреба
cs	Pokyny pro použití
da	Brugsanvisning
de	Gebrauchsanweisung
el	Οδηγίες χρήσης
es	Instrucciones de uso
et	Kasutusjuhised
fi	Käyttöohjeet
fr	Mode d'emploi
hu	Használati utasítás
it	Istruzioni per l'uso
lt	Naudojimo instrukcijos
lv	Lietošanas instrukcija
nl	Gebruiksaanwijzing
no	Bruksanvisning
pl	Instrukcja obsługi
pt	Instruções de utilização
ro	Instrucțiuni de utilizare
sk	Návod na použitie
sv	Bruksanvisning
tr	Kullanım talimatları

**ARGON MEDICAL DEVICES, INC.**

1445 Flat Creek Road
Athens, Texas 75751 USA
Tel: +1 (903) 675 9321
Tel: +1 (800) 927 4669
www.argonmedical.com

**EMERGO EUROPE**

Prinsessegracht 20
2514 AP The Hague
The Netherlands
+31 70 345 8570

The symbols glossary is located electronically at
www.argonmedical.com/symbols

IFU71001/1019C

en - English**Indication for use**

The product is for drainage of cysts, gall bladders and nephrostomies.

Duration

Up to 8 weeks.

Warnings

- This device was designed, tested and manufactured for single patient use only. Reuse or reprocessing of this device may lead to its failure and subsequent patient injury. Reprocessing and/or resterilization of this device may create the risk of contamination and patient infection. Do not reuse, reprocess or resterilize this device.
- Do not use if package appear open or if the expiry date has been exceeded.
- The product must only be used by qualified personnel who are familiar with the technique.

Preparation

- Advance the straightener carefully over the curvature of the catheter, while straightening the curvature with your fingers.
- When fully straighten ensure that the guidewire can pass through the catheter before use.

Procedure

- Advance the catheter over the guidewire that has been introduced into the patient using standard technique.
- When the catheter is fully loaded onto the guidewire remove the straightener.
- Advance the catheter to the drainage site
- Verify correct position of the catheter using fluoroscopy.
- Remove the guidewire.
- Connect the catheter to a drainage bag using a suitable connecting tube.

It is recommended affixing the catheter in a straight line and that any curvature be applied to the connecting tube.

It is recommended that the physician follow the hospital's standard of care procedures for drainage catheters.

Removal of catheter

- Disconnect the drainage bag connector tube from catheter.
- Pull the catheter out gently. If access is to be maintained, a straight floppy tip guidewire passed through the catheter will facilitate removal while maintaining access.

Storage

Store in a dark, dry, cool place. Avoid extended exposure to light.

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