en	Instruction For Use
bg	Инструкции за употреба
cs	Pokyny pro použití
da	Brugsanvisning
de	Gebrauchsanweisung
el	Οδηγίες χρήσης
es	Instrucciones de uso
et	Kasutusjuhised
fi	Käyttöohjeet
fr	Mode d'emploi
hu	Használati utasítás
it	Istruzioni per l'uso
lt	Naudojimo instrukcijos
lv	Lietošanas instrukcija
nl	Gebruiksaanwijzing
no	Bruksanvisning
pl	Instrukcja obsługi
pt	Instruções de utilização
ro	Instrucțiuni de utilizare
sk	Návod na použitie
sv	Bruksanvisning
tr	Kullanım talimatları

ARGON MEDICAL DEVICES, INC.
 1445 Flat Creek Road
 Athens, Texas 75751 USA
 Tel: +1 (903) 675 9321
 Tel: +1 (800) 927 4669
 www.argonmedical.com

EMERGO EUROPE
 Prinsessegracht 20
 2514 AP The Hague
 The Netherlands
 +31 70 345 8570

 The symbols glossary is located at www.argonmedical.com/symbols

CE 2797

IFU7553/0419D

en - English

Indication for use

The product is for biliary drainage

Duration

Up to 12 weeks

Warnings

- This device was designed, tested and manufactured for single use only. Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury. Do not reuse, reprocess or re-sterilize this device.
- Do not use if package appear open or if the expiry date has been exceeded.
- The product must only be used by qualified personnel who are familiar with the technique.

Preparation

- Advance the straightener carefully over the curvature of the catheter, while straightening the curvature with your fingers.
- Gently pull the thread to lock the pigtail in order to avoid unintentional looping of the thread.
- Introduce the chosen stiffener (if plastic: activate the hydrophilic coating with saline) and fasten the stiffener onto the catheter (Luer Lock).
- Remove the straightener
- Activate the coating by saline.

Procedure

- Select and prepare the drainage site using standard technique.
- Insert an appropriate guidewire using standard technique.
- Dilate the tract, if necessary.
- Advance the catheter over the guidewire in fluoroscopy.
- When the catheter tip is inside the biliary tree, loosen the metal stiffener and replace it by the hydrophilic plastic stiffener. Advance the catheter over the guidewire as far as duodenum.
- Remove the stiffener and the guidewire.
- Gently pull the thread turning the catheter counter clockwise to ensure correct position and to complete curvature.
- While pulling the thread tightly fasten the lilac male/female adapter onto the catheter hub. The male luer activates the "string valve".
- Connect the catheter to a drainage bag using a suitable connecting tube.
- Fix the thread tightly by winding it around the slot in the adaptor and finish up with pressing the clip onto the slot and twist off the handle from the clip.
- Verify correct position of the catheter using fluoroscopy.
- The radiopaque marker indicates the last side hole and must be positioned in the biliary passage.

It is recommended affixing the catheter in a straight line and that any curvature be applied to the connecting tube.

It is recommended that the physician follow the hospital's standard of care procedures for drainage catheters.

Removal of catheter

- Disconnect the drainage bag connector tube from catheter.
- Loosen the female/male adaptor from the standard luer hub at the catheter to deactivate the valve.
- Remove the clip and unwind the suture. Check that both threads are loose and cut one thread in order to loosen the pigtail.
- Pull the catheter out gently. If access is to be maintained, a straight floppy tip guidewire passed through the catheter will facilitate removal while maintaining access.

Caution

If the catheter is to be removed on another ward, we recommend that these guidance notes accompany the patient's case notes to ensure that the relevant personnel are aware of the presence of a locking catheter. It is also advisable to inform the patient.

Storage

Store in a dark, dry, cool place. Avoid extended exposure to light.

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