En	Instruction For Use	دواعي الاستعمال
ar		
bg	Инструкции за употреба	
cs	Pokyny pro použití	
da	Brugsanvisning	
de	Gebrauchsanweisung	
el	Οδηγίες χρήσης	
es	Instrucciones de uso	
et	Kasutusjuhised	
fi	Käyttöohjeet	
fr	Mode d'emploi	
hu	Használati utasítás	
it	Istruzioni per l'uso	
lt	Naudojimo instrukcijos	
lv	Lietošanas instrukcija	
nl	Gebbruiksaanwijzing	
no	Bruksanvisning	
pl	Instrukcja obsługi	
pt	Instruções de utilização	
pt-br	Instruções de uso	
ro	Instrucțiuni de utilizare	
sk	Návod na použitie	
sv	Bruksanvisning	
tr	Kullanım talimatları	

**en - English**

**Indication for use**

The guidewire is only to be used for non-vascular procedures.

**Duration**

Transient, less than 60 minutes.

**Warnings**

- This device was designed, tested and manufactured for single use only. Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury. Do not reuse, reprocess or re-sterilize this device.
- Do not use if package appear open or if the expiry date has been exceeded.
- Do not advance the wire against resistance until the cause of the resistance has been determined by fluoroscopy. Excess force against resistance may result in damage to guidewire, or catheter, or vessel perforation.
- Do not withdraw a guidewire through a needle. Straighten the guidewire in order to withdraw the needle.
- The product must only be used by qualified personnel who are familiar with the technique.

**Potential Complications**

Potential complications include, but are not limited to the following:

- Puncture site hematoma
- Infection

**Preparation**

- Make sure that the guidewire is undamaged.
- Loosen the straightener together with the guidewire.
- Inject saline with syringe in a vertical position.

**Storage**

Store in a dark, dry, cool place. Avoid extended exposure to light.


**ARGON MEDICAL DEVICES, INC.**

1445 Flat Creek Road  
Athens, Texas 75751 USA  
Tel: 800-927-4669  
Tel: +1 903-675-9321  
www.argonmedical.com

**EMERGO EUROPE**

Prinsessegracht 20  
2514 AP The Hague  
The Netherlands  
+31 70 345 8570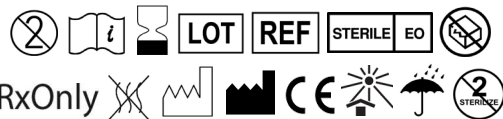

The symbols glossary is located electronically at [www.argonmedical.com/symbols](http://www.argonmedical.com/symbols)

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