en	Instruction For Use	دواعي الاستعمال
ar		
bg	Инструкции за употреба	
cs	Pokyny pro použití	
da	Brugsanvisning	
de	Gebrauchsanweisung	
el	Οδηγίες χρήσης	
es	Instrucciones de uso	
et	Kasutusjuhised	
fi	Käyttöohjeet	
fr	Mode d'emploi	
hu	Használati utasítás	
it	Istruzioni per l'uso	
lt	Naudojimo instrukcijos	
lv	Lietošanas instrukcija	
nl	Gebruiksaanwijzing	
no	Bruksanvisning	
pl	Instrukcja obsługi	
pt	Instruções de utilização	
pt-br	Instruções de uso	
ro	Instrucțiuni de utilizare	
sl	Navodila za uporabo	
sk	Návod na použitie	
sv	Bruksanvisning	
Tr	Kullanım talimatları	
ru	Инструкция по применению	

**ARGON MEDICAL DEVICES, INC.**

1445 Flat Creek Road  
Athens, Texas 75751 USA  
Tel: +1 (903) 675 9321  
Tel: +1 (800) 927 4669  
www.argonmedical.com

**EMERGO EUROPE**

Prinsessegracht 20  
2514 AP The Hague  
The Netherlands  
+31 70 345 8570

The symbols glossary is located electronically at [www.argonmedical.com/symbols](http://www.argonmedical.com/symbols)

IFU6600/0219E

**en - English****Indication for use**

HSG Catheters are intended for use in the injection of contrast material in the examination of the uterus and fallopian tubes.

**Contraindications:** Physician judgment should be used in determining in which patients Hystero-Salpingography or Hystero-Sonography procedures are contraindicated. These procedures are not recommended for patients with active pelvic infections.

**Duration**

Transient, less than 60 minutes.

**Warnings**

- Do not use this device with a power injector for the delivery of the contrast media.
- This device was designed, tested and manufactured for single use only. Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury. Do not reuse, reprocess or re-sterilize this device.
- Do not use if package appear open or if the expiry date has been exceeded.
- Do not use Lipiodol or other oil based contrast media.
- These devices should be used only by a physician familiar with the possible side effects, typical findings, limitations, indications and contraindications of Hystero-Salpingography and Hystero-Sonography.

**Balloon diameter with air:**

- 5.5 F Catheter 11 mm with 2 ml - Accepts 0.035" (0.89mm) guidewire
- 7F Catheter 13 mm with 3 ml - Accepts 0.038" (0.96mm) guidewire

**Balloon diameter with saline:**

- 5.5 F Catheter 10 mm with 0.8 ml - Accepts 0.035" (0.89mm) guidewire
- 7F Catheter 12 mm with 1.2 ml - Accepts 0.038" (0.96mm) guidewire

- When inflating balloon with sterile saline solution (instead of air), do not exceed the volumes specified above.

**Preparation**

- Fill the catheter with contrast liquid and close the stopcock. Remove the contrast syringe to facilitate handling of the catheter.
- Check that the blue stopcock on the balloon port is open and mount the syringe with the required quantity of air (X-ray) or sterile saline (ultrasound).
- If sterile saline is used in the balloon the air is deflated from the system by filling and emptying the balloon with sterile saline before use
- Inflate and deflate the balloon (extend and retract the plunger on the syringe) to insure its functionality and integrity.

**Procedure**

- Insert the introduction tube (IT) into the vagina and guide the balloon catheter through the (IT) into uterus (max 6 cm).
- Inflate the balloon and close the stopcock. Pull back the catheter to secure the balloon thus preventing outflow of contrast liquid into the vagina.
- Fix the introducer tube onto the reinforced part of the catheter shaft at the bifurcation.

- Mount the contrast syringe, open the stopcock, inject contrast liquid and carry out the examination.
- When the examination has been completed, deflate the balloon by opening the balloon port stopcock and drawing back the syringe. Remove the catheter if necessary using contrast injection and fluoroscopy in order to present the cervix uteri.

### Storage

Store in a dark, dry, cool place. Avoid extended exposure to light.

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