**ARGON MEDICAL DEVICES, INC.**  
1445 Flat Creek Road Athens, Texas 75751 USA  
Tel: +1 (903) 675-9321  
Tel: +1 (800) 927-4669  
www.argonmedical.com

**EMERGO EUROPE**  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands  
+31 70 345 8570


**Argon Medical Devices UK Ltd**  
Eastgate Business Centre  
Eastern Avenue  
Burton-on-Trent  
Staffordshire  
DE13 0AT

X9585365/0221A

<https://www.argonmedical.com/resources/product-information>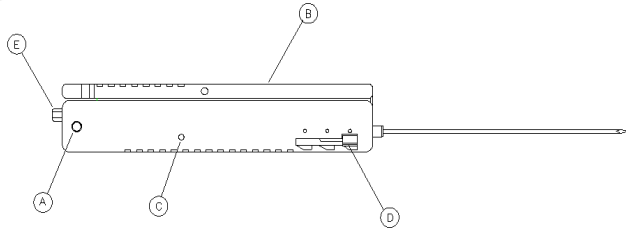

<b>English</b>	The symbols glossary is located electronically at <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Brazilian Portuguese</b>	O glossário dos símbolos está localizado eletronicamente em <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Bulgarian</b>	Речникът на символите е наличен в електронен вид на адрес <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Czech</b>	Slovník symbolů v elektronické formě je umístěn na <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Danish</b>	Symbolforklaringen findes elektronisk på <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>German</b>	Das Glossar der Symbole kann unter <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a> abgerufen werden
<b>Greek</b>	Το γλωσσάρι των συμβόλων παρέχεται ηλεκτρονικά στη διεύθυνση <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Spanish</b>	El glosario de símbolos puede consultarse en línea en la dirección <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Estonian</b>	Sümbolite seletuste kogu on elektrooniliselt saadaval aadressil <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Finnish</b>	Elektroninen symbolisanasto on osoitteessa <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a> .
<b>French</b>	Le glossaire des symboles est disponible au format électronique à l'adresse <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Hungarian</b>	A szimbólumok összefoglalása elektronikusan itt található: <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Italian</b>	Il glossario dei simboli è disponibile in formato elettronico all'indirizzo <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Japanese</b>	記号用語については、電子的手段で <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a> をご確認ください。
<b>Korean</b>	기호의 의미는 <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a> 웹사이트 참조
<b>Lithuanian</b>	Simbolių žodyno elektroninė forma pateikta <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Norwegian</b>	Symbolforklaringen befinner seg elektronisk på <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Dutch</b>	Uitleg over de symbolen is elektronisch beschikbaar op <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>

<b>Polish</b>	Glosariusz symboli w postaci elektronicznej dostępny jest pod adresem <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Portuguese</b>	O glossário de símbolos está localizado eletronicamente no endereço <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Romanian</b>	Glosarul simbolurilor este stocat electronic la <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Russian</b>	Электронный глоссарий символов расположен по адресу <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Slovak</b>	Glosár symbolov je v elektronickej podobe k dispozícii na stránkach <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Slovenian</b>	Slovar simbolov se nahaja v elektronski obliki na <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a> .
<b>Swedish</b>	Symbolforklaringen hittas elektroniskt på <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Turkish</b>	Semboller sözlüğü elektronik ortamda <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a> adresinde yer almaktadır
<b>Simplified Chinese</b>	电子版的符号术语表参见 <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Traditional Chinese</b>	符號詞彙表以電子方式在以下網站提供： <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Arabic</b>	معلومات مفصلة عن الرموز الإلكترونية على <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>

## EN - ENGLISH

### BIOPINCE™ FULL CORE BIOPSY INSTRUMENT

#### INTENDED USE/PURPOSE

BioPince™ Full Core Biopsy Instrument is intended to be used for harvesting multiple core specimens from soft tissue for clinical diagnosis.

#### DEVICE DESCRIPTION:

The instruments are comprised of a lightweight handle with a pre-attached 16ga or 18ga needle set. The device is designed to capture a full core specimen using tri-axial core cut and capture system to harvest diagnostic quality specimens. The device also feature a centimeter markings that provide reference for depth placement. It had a color-coded rear trigger for gauge identification, green ready indicator and forward trigger for flexibility approach.

The variable throw lengths enhance clinical flexibility: 13mm throw yields 9mm specimen length, 23mm throw yields 19mm specimen length, 33mm throw yields 29mm specimen length.

**INDICATIONS FOR USE:** The disposable, automatic BioPince™ Full Core Biopsy Instrument is used to obtain multiple core samples from soft tissue such as liver, kidney, abdominal masses, etc.

**CONTRAINDICATIONS:** Use only for core biopsies of soft tissue as determined by a licensed physician. A physician familiar with the possible side effects, typical findings, limitations, indications and contraindications of core needle biopsy. should use the instrument. Physician judgement is required when considering biopsy on patients with bleeding disorders, or receiving anticoagulant medications.

#### WARNINGS

- Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury. Do not reuse, reprocess or re-sterilize this device.
  - Inspect the package integrity before use.
  - Do not use if package is open or damaged or if the expiry date has been exceeded.
  - The instrument should be used by a physician familiar with the possible side effects, typical findings, limitations, indications and contraindications of core needle biopsy.
  - Use only for core biopsies of soft tissue as determined by a licensed physician. Attempting to biopsy hard tissue or masses may reduce the performance, causing damage to the instrument or potentially harm the patient. This instrument is intended for soft tissue biopsy only.
  - Incorrect setting of stroke length on the device may cause a sample to be incorrectly taken.
  - Handle in a manner, which will prevent accidental puncture.
- Note:** These instructions for the BioPince Full Core Biopsy Instrument are NOT meant to define or suggest any medical or surgical technique. The individual physician is responsible for the proper procedure and techniques to be used with this product.

#### PRECAUTIONS

- The BioPince Full Core Biopsy Instrument is available in different needle lengths and gauges. The appropriate length and gauge of the instrument must be selected for the procedure, per physician judgment.
  - Verify integrity of needle before cocking instrument and after firing(s). If needle is damaged, appropriately discard the entire device (see disposal section) and prepare a new instrument. Do not attempt to open the device.
  - This instrument was designed, tested, and manufactured for single use only.
  - The introduction of the needle into the body should be carried out under imaging control (ultrasound, CT, etc.)
  - This instrument can be used with an appropriate Argon Co-Axial Introducer Needle. Other manufacturer's co-axial introducer needles have not been tested for compatibility with the BioPince Full Core Biopsy Instrument.
  - Do not bend the needle.
  - Do not contact the needle tips to a hard surface while expelling a sample.
- Note:** Deliberate mis-use or off-label use in violation of this IFU may negatively impact the safety of the patient and/or the effectiveness of the device.

#### DIRECTIONS FOR USE:

1. Prepare the biopsy site appropriately and locate the target area.
2. Advance a BioPince™ Co-axial Introducer Needle, using standard technique, to a position proximal to the lesion. Position the adjustable needle stop proximal to the skin.

**Note:** If used without co-axial guidance, an incisional dermatotomy is recommended prior to activating the BioPince device through the skin.

3. Remove the stylet from the Co-axial Introducer Needle, leaving the cannula in position.
4. Activate the safety button (A) on the side of the instrument. It will indicate the "SAFE" position.
5. Cock the instrument by lifting the cocking lever (B) until a "click" is heard. Push the lever down until a second "click" is heard. A YELLOW indicator in the "READY" window (C) indicates that it is cocked.  
Note: BioPince™ has three needle stroke lengths-13, 23, and 33mm to provide clinical flexibility. The instrument is shipped in the 33mm position. If an adjustment to stroke length is desired, the instrument must first be cocked. To select the desired stroke length, lift the "ears" on both sides of the stroke length adjuster (D) and move the adjuster to the appropriate position. The 13mm stroke length yields a specimen of 9mm length, the 23mm stroke length yields a specimen of 19mm length, and the 33mm yields a specimen of 29mm length.
6. Advance the BioPince™ stylet to a position proximal to the lesion and release the safety (A). The instrument is ready to "FIRE".
7. Firmly press the firing trigger (E) located at the proximal end of the instrument.
8. Remove the device from the patient. Lifting the cocking lever (B) and pushing it down will automatically expel the specimen. The instrument is now ready for another specimen retrieval.

#### DISPOSAL

After use, this product may be a potential biohazard. Dispose in accordance with applicable laws and regulations.

#### STORAGE

Store at controlled room temperature.

**NOTE:** In the event a serious incident related to this device occurs, the event should be reported to Argon Medical at [quality.regulatory@argonmedical.com](mailto:quality.regulatory@argonmedical.com) as well as to the competent health authority where the user/patient resides.

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