SKATER™ All-Purpose and Nephrostomy Drainage Set (Locking and Non-Locking Pigtail)

ARGON MEDICAL DEVICES, INC.
1445 Flat Creek Road
Athens, Texas 75751 USA
Tel: +1(903) 675-9321
Tel: +1(800) 927-4669
www.argonmedical.com

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands
+31 70 345 8570

IFU7000CE/1020A

 <https://www.argonmedical.com/resources/product-information>

English	The symbols glossary is located electronically at www.argonmedical.com/symbols
Bulgaria	Речникът на символите е наличен в електронен вид на адрес www.argonmedical.com/symbols
Czech	Slovníček symbolů v elektronické formě je umístěn na www.argonmedical.com/symbols
Danish	Symbolforklaringen findes elektronisk på www.argonmedical.com/symbols
German	Das Symbolglossar ist elektronisch verfügbar unter www.argonmedical.com/symbols
Greek	Το γλωσσάρι των συμβόλων παρέχεται ηλεκτρονικά στη διεύθυνση www.argonmedical.com/symbols
Spanish	El glosario de símbolos puede consultarse electrónicamente en la dirección www.argonmedical.com/symbols
Estonian	Sümbolite sõnastik on elektrooniliselt saadaval aadressil www.argonmedical.com/symbols
Finnish	Symbolisanasto on sähköisessä muodossa osoitteessa www.argonmedical.com/symbols .
French	Le glossaire des symboles est disponible sous forme électronique à l'adresse www.argonmedical.com/symbols
Croatian	Pojmovnik simbola nalazi se u elektroničkom obliku na www.argonmedical.com/symbols
Hungarian	A szimbólumok összefoglalása elektronikusan itt található: www.argonmedical.com/symbols
Italian	Il glossario dei simboli è disponibile in formato elettronico all'indirizzo www.argonmedical.com/symbols
Lithuanian	Simbolių žodynas elektronine forma pateiktas www.argonmedical.com/symbols
Latvian	Simbolu glosārijs ir atrodams elektroniskā veidā vietnē www.argonmedical.com/symbols
Dutch	Uitleg over de symbolen is elektronisch beschikbaar op www.argonmedical.com/symbols
Norwegian	Symbolforklaringen finnes seg elektronisk på www.argonmedical.com/symbols
Polish	Wersja elektroniczna słownika symboli znajduje się na stronie www.argonmedical.com/symbols
Portuguese	O glossário de símbolos está localizado eletronicamente em www.argonmedical.com/symbols
Romanian	Glosarul simbolurilor este stocat electronic la www.argonmedical.com/symbols
Slovenian	Slovar simbolov se nahaja v elektronski obliki na www.argonmedical.com/symbols
Slovak	Glosár symbolov je v elektronickej podobe k dispozícii na stránkach www.argonmedical.com/symbols
Swedish	Symbolordlistan finns elektroniskt på www.argonmedical.com/symbols
Turkish	Semboller sözlüğü elektronik ortamda www.argonmedical.com/symbols adresinde yer almaktadır

EN-English

SKATER™ All-Purpose and Nephrostomy Drainage Set (Locking and Non-Locking Pigtail)

Intended Use/Purpose

The SKATER™ All-Purpose and Nephrostomy Drainage Set is used for fluid drainage procedures.

Indication for use

The product is for percutaneous drainage in a variety of drainage applications (e.g., nephrostomy and abscess).

Device Description

The SKATER™ All-Purpose and Nephrostostomy Drainage Set Catheters are designed to promote maximum drainage flow. They are formulated for patient comfort, kink resistance, and easy insertion.

Duration

Up to 12 weeks.

Warnings

- This device was designed, tested and supplied sterile and is intended for single use only. Do not re-sterilize.
- Do not reuse or reprocess this device. Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury.
- Inspect the package integrity before use.
- Do not use if package is open or damaged, or if the expiration date has been exceeded.
- During insertion, avoid contact with bone, cartilage, and scar tissue which can damage the catheter tip.

Caution (Locking Pigtail): If the catheter is to be removed in another department, we recommend that these guidance notes accompany the patient's case notes to ensure that the relevant personnel are aware of the presence of a locking catheter. It is also advisable to inform the patient.

Precautions

- The product must only be used by licensed, knowledgeable physician and qualified personnel who are familiar with the technique and who can ensure that patients will be monitored and appropriately treated for expected complications.
- Ensure the drain is secured and the system is intact to prevent dislodgement. Secure with a catheter fixation device, suture, or tape.
- Assess drain insertion site for signs of leakage, redness, or oozing. These signs may indicate an infection or irritation of the surrounding skin.
- Monitor changes in character or volume of fluid or bleeding.
- It is recommended affixing the catheter in a straight line and that any curvature be applied to the connecting tube.
- The drain also provides a pathway for bacteria to get into the wound. The risk of infection can be due to ascending bacterial invasion, foreign body reaction, decreased local tissue resistance, bacterial hiding places, poor placement due to kinked drain, and poor postoperative management.
- It is recommended that the physician follow the hospital's standard of care procedures for drainage catheters.
- As with any draining procedure, there is risk of infection, pain, discomfort, inadvertent tissue damage, and fluid loss. When used as intended by a physician, the benefits of using the SKATER™ All-Purpose and Nephrostomy Drainage Set outweigh the risks associated with the use of the device.

Preparation

- Advance the straightener carefully over the curvature of the catheter while straightening the curvature with your fingers.
- Locking Pigtail** - Gently pull the thread to lock the pigtail in order to avoid unintentional looping of the thread.
- For Single-Step:**
 - Introduce the metal stiffener fully with the trocar needle slightly drawn back to avoid damaging the catheter wall.
 - Fasten the stiffener onto the catheter and introduce the trocar needle. Support the tip of the catheter advancing the needle. Snap the Choice Lock™ onto the stiffener hub.
NOTE: 35, 45, and 60cm devices are not provided with a trocar needle and must be placed over the wire. Guidewire not included.
- For Over the Wire:**
 - Introduce the chosen stiffener and fasten the stiffener onto the catheter (Luer Lock).
NOTE: The flexible stiffener included with 45 and 60cm devices is coated. Activate the coating with saline prior to use.
- Remove the straightener and the catheter is now ready for use.
- Activate the catheter coating by saline.

Procedure

- For Single-Step:**
 - Perform the skin incision under local anaesthesia.

2. Introduce the catheter set to the cavity using ultrasound guidance, CT or fluoroscopy.
 3. Verify correct position by releasing the Choice Lock™ with one hand and remove the stylet. Check that fluid is escaping or aspirate with a 10ml syringe.
 4. If you want to change the placement, the stylet must always be correctly positioned.
 5. When correctly positioned, advance the catheter over the needle, holding the needle steady.
- **For Over the Wire:**
 1. Select and prepare the drainage site using standard technique.
 2. Insert an appropriate guidewire using standard technique.
 3. Dilate the tract, if necessary.
 4. Under fluoroscopic guidance, advance the catheter over the guidewire.
 5. When the catheter tip is into the cavity, loosen the stiffener.
 6. Hold the stiffener tight and advance the catheter over the stiffener until the complete length of the pigtail is in the cavity.
 7. Remove the stiffener and the guidewire.
 - Ensure that the complete length of the pigtail is in the cavity.
 - **Non-Locking Pigtail:** The pigtail "curls" in the cavity. Remove the needle and connect the catheter to a drainage bag using a suitable connecting tube.
 - **Locking Pigtail:**
 1. Gently pull the thread to ensure correct position and to complete curvature of the pigtail.
 2. Fix the thread tightly by winding it around the slot in the hub and finish by pressing the clip-on top of the slot and twist off the handle from the clip.
 3. Connect the catheter to a drainage bag using a suitable connecting tube.

Removal of catheter

- Disconnect the drainage bag connector tube from catheter.
- **Locking Pigtail** - Remove the clip and unwind the suture. Check that both threads are loose and cut one thread in order to loosen the pigtail.
- Pull the catheter out gently. If access is to be maintained, a straight floppy tip guidewire passed through the catheter will facilitate removal while maintaining access.

Disposal

After use, handle and dispose in accordance with hospital policies and procedures concerning biohazard materials and waste.

Storage

Store in controlled room temperature.

NOTE: In the event a serious incident related to this device occurs, the event should be reported to Argon Medical at quality.regulatory@argonmedical.com as well as to the competent health authority where the user/patient resides.

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