**ARGON MEDICAL DEVICES, INC.**

1445 Flat Creek Road  
Athens, Texas 75751 USA  
Tel: +1 800-927-4669  
Tel: +1 903-675-9321  
www.argonmedical.com

**EMERGO EUROPE**  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands  
+31 70 345 8570

**Argon Medical Devices UK Ltd**

Eastgate Business Centre  
Eastern Avenue  
Burton-on-Trent  
Staffordshire  
DE13 0AT

IFU9009/0221A

<https://www.argonmedical.com/resources/product-information>

English	The symbols glossary is located electronically at <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
Български	Речникът на символите е наличен в електронен вид на адрес <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
Česky	Slovník symbolů v elektronické formě je umístěn na <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
Dansk	Symbolforklaringen findes i elektronisk form på <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
Deutsch	Das Glossar der Symbole kann unter <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a> abgerufen werden
Ελληνικά	Το γλωσσάρι των συμβόλων παρέχεται ηλεκτρονικά στη διεύθυνση <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
Español	El glosario de símbolos puede consultarse en línea en la dirección <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
Eesti	Sümbolite seletuste kogu on elektrooniliselt saadaval aadressil <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
Suomi	Elektroninen symbolisanasto on osoitteessa <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a> .
Français	Le glossaire des symboles est disponible au format électronique à l'adresse <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
Magyar	A szimbólumok összefoglalása elektronikusan itt található: <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
Italiano	Il glossario dei simboli è disponibile in formato elettronico all'indirizzo <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
Lietuvių k.	Simbolių žodyno elektroninė forma pateikta <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
Latviešu val.	Simbolu glosārijs ir atrodams elektroniskā veidā vietnē <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
Nederlands	Uitleg over de symbolen is elektronisch beschikbaar op <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
Norsk	Symbolforklaringen befinner seg elektronisk på <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
Polski	Glosariusz symboli w postaci elektronicznej dostępny jest pod adresem <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
Português	O glossário de símbolos está localizado eletronicamente no endereço <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
Română	Glosarul simbolurilor este stocat electronic la <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
Slovenčina	Glosár symbolov je v elektronickej podobe k dispozícii na stránkach <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
Svenska	Symbolforklaringen hittas elektroniskt på <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
Türkçe	Semboller sözlüğü elektronik ortamda <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a> adresinde yer almaktadır

**Intended Use/Purpose**

The SKATER™ Single Step Drainage Set is used for percutaneous drainage procedures.

**Device Description**

The SKATER™ Single Step Drainage Set are designed to promote maximum drainage flow. They are formulated for patient comfort, kink resistance, and easy insertion.

**Indication for use**

**Non-Locking Pigtail** - the product is for single step drainage of cysts, abscesses, hematomaes, pleural exudates, ascites, gall bladders and nephrostomies.

**Locking Pigtail** - The product is for single step drainage of cysts, gall bladders and nephrostomies.

**Duration**

Up to 12 weeks

**Warnings**

- This device was designed, tested and manufactured for single use only. Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury. Do not reuse, reprocess or re-sterilize this device.
- Inspect the package integrity before use.
- Do not use if package appear open or if the expiry date has been exceeded.
- During insertion avoid contact with bone, cartilage and scar tissue which can damage the catheter tip.

**Caution (Locking Pigtail)**

If the catheter is to be removed in another department, we recommend that these guidance notes accompany the patient's case notes to ensure that the relevant personnel are aware of the presence of a locking catheter. It is also advisable to inform the patient.

**Precaution**

- The product must only be used by licensed, knowledgeable physician and qualified personnel who are familiar with the technique and who can ensure that patients will be monitored and appropriately treated for expected complications.
- Ensure the drain is secured and the system is intact to prevent dislodgement. Secure with a catheter fixation device, suture, or tape.
- Assess drain insertion site for signs of leakage, redness, or oozing. These signs may indicate an infection or irritation of the surrounding skin.
- Monitor changes in character or volume of fluid or bleeding.
- It is recommended affixing the catheter in a straight line and that any curvature be applied to the connecting tube.
- The drain also provides a pathway for bacteria to get into the wound. The risk of infection can be due to ascending bacterial invasion, foreign body reaction, decreased local tissue resistance, bacterial hiding places, poor placement due to kinked drain, and poor postoperative management.
- It is recommended that the physician follow the hospital's standard of care procedures for drainage catheters.
- As with any draining procedure, there is risk of infection, pain, discomfort, inadvertent tissue damage, and fluid loss. When used as intended by a physician, the benefits of using the SKATER™ Single Step Drainage Catheter outweigh the risks associated with the use of the device.

**Preparation**

- Advance the straightener carefully over the curvature of the catheter, while straightening the curvature with your fingers.
- Locking Pigtail** - Gently pull the thread to avoid unintentional looping of the thread.
- Introduce the metal stiffener fully with the trocar needle slightly drawn back to avoid damaging the catheter wall.
- Fasten the stiffener onto the catheter and introduce the trocar needle. Support the tip of the catheter advancing the needle. Snap the Choice Lock™ onto the stiffener hub.
- Remove the straightener and the catheter is now ready for use.
- Activate the coating by saline.

**Procedure**

**Single-step catheterisation through a skin incision.**

- Perform the skin incision under local anaesthesia.
- Introduce the catheter set to the cavity using ultrasound guidance, CT or fluoroscopy.
- Verify correct position by releasing the Choice Lock™ with one hand and remove the stylet. Check that fluid is escaping or aspirate with a 10 ml syringe. The stylet must always be correctly positioned if you want to change the placement. When correctly positioned advance the catheter over the needle, holding the needle steady.
- Ensure that the complete length of the pigtail is in the cavity.
- Non-Locking Pigtail**
  - The pigtail "curls" in the cavity. Remove the needle and connect the catheter to a drainage bag with suitable connecting tube.
- Locking Pigtail**
  - Gently pull the thread to ensure correct position and to complete curvature of the pigtail.

- Fix the thread tightly by winding it around the slot in the hub and finish by pressing the clip onto the slot and twist off the handle from the clip.
- Connect the catheter to a drainage bag using a suitable connecting tube.

#### Removal of catheter

- Disconnect the drainage bag connector tube from catheter.
- Locking Pigtail** - Remove the clip and unwind the suture. Check that both threads are loose and cut one thread in order to loosen the pigtail.
- Pull the catheter out gently. If access is to be maintained, a straight floppy tip guidewire passed through the catheter will facilitate removal while maintaining access.

#### Disposal

After use, handle and dispose in accordance with hospitals policies and procedures concerning biohazard materials and waste.

#### Storage

Store in controlled room temperature.

**NOTE:** In the event a serious incident related to this device occurs, the event should be reported to Argon Medical at [quality.regulatory@argonmedical.com](mailto:quality.regulatory@argonmedical.com) as well as to the competent health authority where the user/patient resides.

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