**ARGON MEDICAL DEVICES, INC.**  
1445 Flat Creek Road Athens, Texas 75751 USA  
Tel: +1 800-927-4669;  
Tel: +1 903-675-9321  
www.argonmedical.com


**EMERGO EUROPE**  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands  
+31 70 345 8570

**Argon Medical Devices UK Ltd**  
Eastgate Business Centre  
Eastern Avenue  
Burton-on-Trent  
Staffordshire  
DE13 0AT

X9585367/0321A

BARD® and MAGNUM® are registered trademarks of C.R. Bard, Inc., or an affiliate

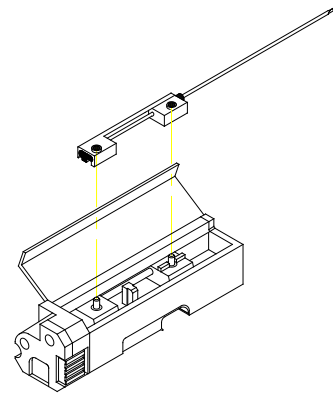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## UltraCore™ Biopsy Needle

### ENGLISH

**Intended Use/Purpose:** This automatic cutting needle is intended for use as a core biopsy needle and is to be used in conjunction with the Bard® Magnum® and BIP High Speed Multi Biopsy Instrument or other compatible, commercially available biopsy instruments.

**Description:** This device features a high quality core biopsy needle that provides histological specimens for diagnosis of suspect lesions. Each needle has an echogenic tip, promoting accurate placement under ultrasound guidance and numerical order centimeter markings that facilitate depth placement. The hubs are color coded for easy gauge size determination and it can be used with echogenic co-axial introducer needles.

**Indications for Use:** The UltraCore™ Biopsy Needle is used to obtain multiple core samples from soft tissue such as the liver, kidney, prostate, breast and various soft tissue lesions. It is not intended for bone.

**Contraindications:** For use only for core biopsies of soft tissue as determined by a licensed physician. The instrument should be used by a physician familiar with the possible side effects, typical findings, limitations, indications and contraindications of core needle biopsy. Physician judgment is required when considering biopsy on patients with a bleeding disorder or receiving anti-coagulant medications.

#### Warning:

- This device was designed, tested and manufactured for single use only. Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury. Do not reuse, reprocess or re-sterilize this device.
- Inspect the package integrity before use.
- Do not use if package is open or damaged and if the expiry date has been exceeded.
- Do not continue to use if any of the component are damaged during the procedure.

#### Precaution:

- This device is intended for use by trained qualified physician.
- Failure to align biopsy device pegs to the needle holes when loading will cause needle malfunction.

#### Pre-procedure Preparation/ Directions for Use:

- Inspect the product package for damage. If undamaged, open the package using proper aseptic technique.
- Remove the needle from the pouch and remove the tip protector. Inspect the stylet and cannula for damage or imperfections that would prevent proper operation of the needle assembly.
- Open the lid of the biopsy device and load the needle set into the device as illustrated. The needle set may be loaded in either cocked or uncocked position.
- Operate the biopsy device following the manufacturer's operating instructions.

#### Possible Complications:

- Hemorrhage
- Infection
- Adjacent Tissue Injury
- Pain
- Hemoptysis
- Hemothorax
- Organ or Vessel Perforation
- Pneumothorax
- Air Embolism

**Disposal:** After use, this product may be a potential biohazard. Handle in a manner which will prevent accidental puncture. Dispose in accordance with applicable laws and regulations.

**Storage:** Store in controlled room temperature.

**NOTE:** In the event a serious incident related to this device occurs, the event should be reported to Argon Medical at [quality.regulatory@argonmedical.com](mailto:quality.regulatory@argonmedical.com) as well as to the competent health authority where the user/patient resides.

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