
ARGON

MEDICAL DEVICES

Splittable Needle Introducer

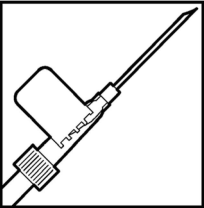STERILE

EO

Rx ONLY

en	Splittable Needle Introducer
----	------------------------------

GENERAL GUIDELINES:

- The Argon Splittable Needle Introducer has been designed for use with Argon L-Cath™ catheter products.
- For proper use, clinicians must be familiar with and trained in the use of these products. Use of these devices should be preceded by an established institutional protocol and performed by persons trained in the procedure and knowledgeable of the inherent risks. Argon provides a comprehensive clinical education program to instruct clinicians on recommended practices for catheter insertion and maintenance.
- REPORT NEEDLESTICKS IMMEDIATELY AND FOLLOW ESTABLISHED PROTOCOL. Percutaneous puncture with a contaminated needle may lead to serious illness such as hepatitis, HIV (AIDS) or other infectious diseases. Resheathing needles is hazardous.
- Refer to appropriate institutional or governmental guidelines for handling and discarding needles and other sharps as well as proper disposal of all potentially contaminated items.
- DO NOT reuse introducer after a failed attempt.
- DEHP free.
- This product does not contain natural rubber latex.

Instructions for Use

- Remove protective cover.
- Grasp the needle wing assembly or flash chamber and perform venipuncture.
- Confirm blood return and release tourniquet. Stabilize needle then remove flash chamber.
- Grasp the catheter close to the tip and advance approximately 10cm. **CAUTION: Do not grasp the catheter tightly with forceps. CAUTION: NEVER pull the catheter back through the needle as this could cut or sever the catheter.** If necessary, withdraw catheter and needle back together.
- Maintain the catheter position by applying digital pressure to the vein above the introducer. Slowly withdraw the needle from the vein and away from the site. CAUTION: Keep the needle parallel to the catheter to prevent damage.
- Press the wings together smartly with the thumb and forefinger until you hear a snap. Grasp the wings, bend them down and peel needle apart, using a forward motion. Use care to maintain catheter position.
- Alternate method - Rotate needle so wings point down. Apply pressure upward on wings and hub to snap. Pull apart needle with a forward motion.

es	Introductor de aguja divisible
----	--------------------------------

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Brasil Detentor de Registro: CM HOSPITALAR S.A.