<b>Simplified Chinese</b>	电子版的符号术语表参见 <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Traditional Chinese</b>	符號詞彙表以電子方式在以下網站提供： <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Arabic</b>	يوفر مصدر الرموز الإلكتروني على <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>

**ARGON MEDICAL DEVICES, INC.**  
1445 Flat Creek Road Athens, Texas 75751 USA  
Tel: +1 (903) 675 9321  
Tel: +1 (800) 927 4669  
[www.argonmedical.com](http://www.argonmedical.com)

**EMERGO EUROPE**  
Westervoortsedijk 60  
6827 AT Arnhem  
The Netherlands  
+31 70 345 8570

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**Emergo Consulting (UK) Limited**  
c/o Cr 360 - UL International  
Compass House, Vision Park Histon  
Cambridge CB24 9BZ  
United Kingdom

**ENGLISH**

**BioPince™ Ultra Full Core Biopsy Instrument**

**INTENDED USE/PURPOSE:** BioPince™ Ultra Full Core Biopsy Instrument is intended to be used for harvesting multiple core specimens from soft tissue for clinical diagnosis.

**INDICATIONS FOR USE:** The disposable, automatic BioPince Ultra Full Core Biopsy Instrument is used to obtain multiple core samples from soft tissue such as liver, kidney, abdominal masses, etc.

**DEVICE DESCRIPTION:** The BioPince Ultra Full Core Biopsy Instrument is a disposable, automatic biopsy instrument that cuts a full core sample of soft tissue. The instrument is designed to expel the sample upon re-cocking of the instrument which prepares the instrument for taking another sample (i.e., ready to be fired). The instrument has a safety slide, a ready indicator window and two firing trigger buttons. Additionally, each instrument has an adjustable stroke length (i.e., throw) of three settings:

Instrument Stroke Length Setting	Approximate Sample Length
13mm	9mm
23mm	19mm
33mm	29mm

The instrument is available in 16ga in lengths of 10cm or 15cm; and 18ga in lengths of 10cm, 15cm or 20cm. The instrument can be used with a corresponding co-axial introducer needle.

**HOW SUPPLIED:**


This product is supplied sterile unless the package has been opened or damaged. Sterilized by Ethylene Oxide (EO). For Single Use Only. Do Not Resterilize.

**CONTRAINDICATIONS:** Physician judgement is required when considering biopsy on patients with bleeding disorders or receiving anticoagulant medications.

**WARNINGS:**

- Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury. DO NOT reuse, reprocess or re-sterilize this instrument.
- Inspect the package integrity before use.
- Do not use if package is open or damaged or if the expiry date has been exceeded.
- The instrument should be used by a physician familiar with the possible side effects, typical findings, limitations, indications and contraindications of core needle biopsy.
- Use only for core biopsies of soft tissue as determined by a licensed physician. Attempting to biopsy hard tissue or masses may reduce the performance, causing damage to the instrument or potentially harm the patient. This instrument is intended for soft tissue biopsy only.
- Incorrect setting of stroke length on the device may cause a sample to be incorrectly taken.
- Handle in a manner, which will prevent accidental puncture.
- Failure to properly set the device before use may result in excessive travel that may result in patient injury or death.

**Note:** These instructions for the BioPince Ultra Full Core Biopsy Instrument are NOT meant to define or suggest any medical or surgical technique. The individual physician is responsible for the proper procedure and techniques to be used with this product.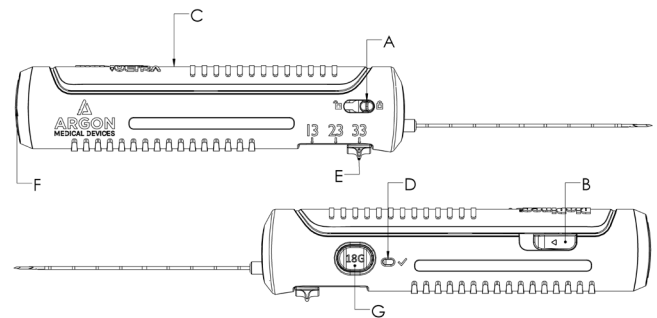

**PRECAUTIONS:**

- The BioPince Ultra Full Core Biopsy Instrument is available in different needle lengths and gauges. The appropriate length and gauge of the instrument must be selected for the procedure, per physician judgment.
- This instrument was designed, tested, and manufactured for single use only.
- Do not bend the needle.
- Do not contact the needle tips to a hard surface while expelling a sample.
- Verify integrity of needle before cocking instrument and after firing(s). If needle is damaged, appropriately discard the entire instrument (see Disposal section) and prepare a new instrument. Do not attempt to open the instrument.
- The introduction of the needle into the body should be carried out under imaging control (ultrasound, CT, etc.)
- This instrument can be used with an appropriate Argon Co-Axial Introducer Needle. Other manufacturer's co-axial introducer needles have not been tested for compatibility with the BioPince Ultra Full Core Biopsy Instrument.

**Note:** Deliberate mis-use or off-label use in violation of this IFU may negatively impact the safety of the patient and/or the effectiveness of the device.


<https://www.argonmedical.com/resources/product-information>

<b>English</b>	The symbols glossary is located electronically at <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Bulgarian</b>	Речникът на символите е наличен в електронен вид на адрес <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Czech</b>	Slovník symbolů v elektronické formě je umístěn na <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Danish</b>	Symbolforklaringen findes elektronisk på <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>German</b>	Das Glossar der Symbole kann unter <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a> abgerufen werden
<b>Greek</b>	Το γλωσσάρι των συμβόλων παρέχεται ηλεκτρονικά στη διεύθυνση <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Spanish</b>	El glosario de símbolos puede consultarse en línea en la dirección <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Estonian</b>	Sümbolite seletuste kogu on elektrooniliselt saadaval aadressil <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Finnish</b>	Elektroninen symbolisanasto on osoitteessa <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>French</b>	Le glossaire des symboles est disponible au format électronique à l'adresse <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Hungarian</b>	A szimbólumok összefoglalása elektronikusan itt található: <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Italian</b>	Il glossario dei simboli è disponibile in formato elettronico all'indirizzo <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Japanese</b>	記号用語については、電子的手段で <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a> をご確認ください。
<b>Korean</b>	기호의 의미는 <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a> 웹사이트 참조
<b>Lithuanian</b>	Simbolių žodyno elektroninė forma pateikta <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Norwegian</b>	Symbolforklaringen finnes seg elektronisk på <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Dutch</b>	Uitleg over de symbolen is elektronisch beschikbaar op <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Polish</b>	Głosariusz symboli w postaci elektronicznej dostępny jest pod adresem <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Portuguese</b>	O glossário de símbolos está localizado eletronicamente no endereço <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Romanian</b>	Glosarul simbolurilor este stocat electronic la <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Russian</b>	Glosarul simbolurilor este stocat electronic la <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Slovak</b>	Glosár symbolov je v elektronickej podobe k dispozícii na stránkach <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Slovenian</b>	Slovar simbolov se nahaja v elektronski obliki na <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Swedish</b>	Symbolforklaringen hittas elektroniskt på <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a>
<b>Turkish</b>	Semboller sözlüğü elektronik ortamda <a href="http://www.argonmedical.com/symbols">www.argonmedical.com/symbols</a> adresinde yer almaktadır

#### DIRECTIONS FOR USE:


1. Prepare the biopsy site appropriately and locate the target area.
2. Using standard technique, advance the co-axial introducer needle, proximal to the lesion. Adjust the needle stop proximal to the skin.  
**Note:** If used without co-axial guidance, an incisional dermatotomy is recommended prior to advancing the BioPince Ultra instrument through the skin.
3. Remove the stylet from the co-axial introducer needle, leaving the cannula in position, if co-axial guidance is used.
4. Engage the safety slide (A) on the side of the instrument to place in the locked position, if desired.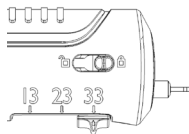

5. Cock the instrument by first pushing forward on the release slide (B) until the cocking lever (C) raises, and then lift the cocking lever (C) until a click is heard. Push the cocking lever down until a second click is heard. Green in the ready window (D) indicates that the instrument is cocked.
6. The instrument is shipped in the 33mm stroke length position. To adjust the stroke length, push in the stroke length adjuster (E) and slide the stroke length adjuster until the arrow on the stroke length adjuster aligns with the arrow of the desired stroke length on the instrument.

**If properly aligned, the user will feel the stroked adjuster slide into the detent. There is also an audible sound when the detent is engaged.**

**WARNING: Failure to align with detent may result in severe patient injury. If the arrow of the stroke length adjuster is not aligned with the arrow for the specific desired value (13, 23, 33) it is NOT in the detent, and the needle will travel a greater distance than intended.**

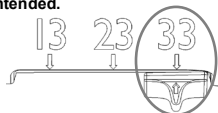- Note:** If an adjustment to the stroke length is desired, the instrument must first be cocked.
7. Advance the BioPince Ultra needle into the co-axial introducer needle or percutaneously to a position proximal to the area to be sampled.
  8. If the instrument is locked, move the safety slide (A) to place in the un-locked position until green is shown in the window. The instrument is ready to fire.
  9. Firmly press on one of the firing trigger buttons located at the proximal end of the instrument (F) or on the side of the instrument (G).
  10. Remove the instrument from the patient.
  11. To expel the sample, push forward on the release slide (B) until the cocking lever (C) raises. Lift the cocking lever (C) until a click is heard, then push the cocking lever down until a second click is heard.
  12. The instrument is cocked and ready for collecting another sample, if additional biopsies are required. Verify integrity of needle tips before additional biopsies.

#### DISPOSAL

After use, this product may be a potential biohazard. Dispose in accordance with applicable laws and regulations.

#### STORAGE

Store at controlled room temperature.

**NOTE:** In the event a serious incident related to this device occurs, the event should be reported to Argon Medical at [quality.regulatory@argonmedical.com](mailto:quality.regulatory@argonmedical.com) as well as to the competent health authority where the user/patient resides.

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