ARGON MEDICAL DEVICES, INC.
1445 Flat Creek Road
Athens, Texas 75751 USA
Tel: +1 800-927-4669;
Tel: +1 903-675-9321
www.argonmedical.com

EC REP **EMERGO EUROPE**
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands
+31 70 345 8570

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UK REP **Emergo Consulting (UK) Limited**
c/o Cr 360 - UL International
Compass House, Vision Park Histon
Cambridge CB24 9BZ
United Kingdom

IFU9013
Date of Issue: 08/18/2023 Rev. B

<https://www.argonmedical.com/resources/product-information>

| | |
|-------------------|--|
| English | The symbols glossary is located electronically at www.argonmedical.com/symbols |
| Bulgarian | Речникът на символите е наличен в електронен вид на адрес www.argonmedical.com/symbols |
| Czech | Slovníček symbolů je elektronicky umístěn na www.argonmedical.com/symbols |
| Danish | Symbolforklaringen findes elektronisk på www.argonmedical.com/symbols |
| German | Ein elektronisches Symbolglossar finden Sie unter www.argonmedical.com/symbols |
| Greek | Το γλωσσάρι των συμβόλων παρέχεται ηλεκτρονικά στη διεύθυνση www.argonmedical.com/symbols |
| Spanish | El glosario de símbolos puede consultarse electrónicamente en www.argonmedical.com/symbols |
| Estonian | Sümbolite seletuste kogu on elektrooniliselt saadaval aadressil www.argonmedical.com/symbols |
| Finnish | Elektroninen symbolisanasto on osoitteessa www.argonmedical.com/symbols . |
| French | Le glossaire des symboles est disponible au format électronique à l'adresse www.argonmedical.com/symbols |
| Croatian | Pojmovnik simbola nalazi se u elektroničkom obliku na www.argonmedical.com/symbols |
| Hungarian | A szimbólumok összefoglalása elektronikusan itt található: www.argonmedical.com/symbols |
| Italian | Il glossario dei simboli è disponibile in formato elettronico all'indirizzo www.argonmedical.com/symbols |
| Lithuanian | Simbolių žodyno elektroninė forma pateikta www.argonmedical.com/symbols |
| Latvian | Simbolu glosārijs ir atrodams elektroniskā veidā vietnē www.argonmedical.com/symbols |
| Dutch | Uitleg over de symbolen is elektronisch beschikbaar op www.argonmedical.com/symbols |
| Norwegian | Symbolforklaringen befinner seg elektronisk på www.argonmedical.com/symbols |
| Polish | Glosariusz symboli w postaci elektronicznej dostępny jest pod adresem www.argonmedical.com/symbols |
| Portuguese | O glossário de símbolos está localizado eletronicamente no endereço www.argonmedical.com/symbols |
| Romanian | Glosarul simbolurilor este stocat electronic la www.argonmedical.com/symbols |
| Slovak | Glossár symbolov je v elektronickej podobe k dispozícii na stránkach www.argonmedical.com/symbols |
| Slovenian | Slovar simbolov se nahaja v elektronski obliki na www.argonmedical.com/symbols . |
| Swedish | Symbolforklaringen hittas elektroniskt på www.argonmedical.com/symbols |
| Turkish | Semboller sözlüğü elektronik ortamda www.argonmedical.com/symbols adresinde yer almaktadır |

ENGLISH

Guidewire Introducer Needles / Seldinger Needle / Potts-Courmand Needle

Intended Use/Purpose

These devices are intended for percutaneous introduction and placement of catheters and guidewires.

Devices Description

These products features a tapered luer hub that allows for easy guidewire insertion. The product offers with a clear window wall hub and winged base plate. They are available in 18G and 19G with lengths of 7cm and 5.7cm. The 18G needle accepts a 0.038" guidewire and the 19G needle accepts a 0.035" guidewire.

Indication for Use

The **Guidewire Introducer** is intended for use as an anterior, single wall percutaneous puncture. The **Seldinger Needle** is intended for use for atraumatic percutaneous introduction of arterial pressure monitoring. The **Potts-Courmand Needle** is intended for use for introduction of devices for carotid angiography, direct arterial pressure monitoring, blood sampling, and percutaneous catheterization.

Warnings :

- Contents are supplied sterile and are intended for single use only. Do not re-sterilize, reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury.
- Inspect the package integrity before use.
- Do not use if package is open or damaged and if the expiry date has been exceeded.
- Do not continue to use if any of the component are damaged during the procedure.

Precautions

- This device is intended for use by trained, qualified licensed physician.

Pre-Procedural Preparations : Flush needles with sterile saline.

Direction for Use :

- Prepare and drape the skin at the intended puncture site; achieve local anesthesia as necessary.
- Using imaging guidance insert the needle through the skin
- Using imaging guidance advance the needle to the intended vessel or target site.
- Advance the appropriate guidewire of choice through the needle.
- Confirm access to desired location with wire
- Remove needle and continue with procedure.

Possible complications :

- Minor Pain at insertion site
- Minor Bleeding
- Hematoma and insertion site
- Infection
- Vessel puncture

Storage: Store at controlled room temperature.

Disposal: After use, this product may be a potential biohazard. Handle in a manner which will prevent accidental puncture. Dispose in accordance with applicable laws and regulations.

NOTE: In the event a serious incident related to this device occurs, the event should be reported to Argon Medical at quality.regulatory@argonmedical.com as well as to the competent health authority where the user/patient resides.

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