ARGON MEDICAL DEVICES, INC.
1445 Flat Creek Road, Athens, Texas 75751 USA
Tel: 800-927-4669;
Tel: +1 903-675-9321
www.argonmedical.com

STERILE EO

Eng: Hystero-Salpingography (HSG) Catheter

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Por: Cateter De Histero-Salpingografia (HSG)

ENG:

Indications for Use: HSG Catheters are intended for use in the injection of contrast material in the examination of the uterus and fallopian tubes.

Contraindications: Physician judgment should be used in determining in which patients Hystero-Salpingography or Hystero-Sonography procedures are contraindicated. These procedures are not recommended for patients with active pelvic infections.

Potential Complications include:

- Bleeding

Warnings:

- Do not use Lipiodol or other oil based contrast media.
- Do not use this device with a power injector for the delivery of the contrast media.
- These devices should be used only by a physician familiar with the possible side effects, typical findings, limitations, indications and contraindications of Hystero-Salpingography and Hystero-Sonography.
- For Single Patient Use Only.** Do not attempt to clean or resterilize this product. After use, this product may be a potential biohazard. Dispose in accordance with applicable laws and regulations.

Preparations:

- Refer to the following chart to aid in selecting the proper catheter:

Catalog No.	Accepts GW	Balloon Dia. W/ air	Balloon Dia. W/ saline
660005040 (5.5Fr)	0.035" (0.89mm)	11mm with 2.0ml air	10mm with 0.8ml saline
660007040 (7Fr)	0.038" (0.96mm)	13mm with 3.0ml air	12mm with 1.2ml saline

- Check to insure that both the blue stopcock (balloon port) and the white stopcock (contrast media port) operate freely. Set both stopcocks to the open position.
- Inflate and deflate the balloon (extend and retract the plunger on the vented syringe) to insure its functionality and integrity.

NOTE: When inflating balloon with saline solution (instead of air), do not exceed the volumes specified in the above table.

- Expel air from the catheter by filling it with contrast media through the white stopcock. Close the white stopcock.
- Remove the contrast media syringe to facilitate easier procedural handling.

Instructions:

- Advance the introduction tube forward until the distal end of the catheter protrudes slightly from the distal end of the sheath.
- Visualize the external cervical Os with the aid of a speculum and advance the introduction tube/catheter such that the tip of the catheter enters the cervical canal.
- Holding the introduction tube stationary, advance the catheter into the cervical canal 1-2 cm.
- Slowly** inflate the balloon, providing occlusion, without causing patient discomfort.
- If uterine placement is desired, advance the catheter into the uterus, inflate the balloon and gently withdraw against the internal Os.
- Attach the contrast media syringe, open the white stopcock, inject the contrast liquid and conduct the examination.
- When the examination has been completed, deflate the balloon by opening the blue stopcock and retracting the syringe plunger.
- Remove the catheter and introduction tube.

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