ARGON MEDICAL DEVICES, INC.

1445 Flat Creek Road
Athens, Texas 75751 USA
Tel: +1 (903) 675 9321
Tel: +1 (800) 927 4669
www.argonmedical.com

EMERGO EUROPE

Prinsessegracht 20
2514 AP The Hague
The Netherlands
+31 70 345 8570

Argon Medical Devices UK Ltd

Eastgate Business Centre
Eastern Avenue
Burton-on-Trent
Staffordshire
DE13 0AT

IFU9007/0221A

<https://www.argonmedical.com/resources/product-information>

English	The symbols glossary is located electronically at www.argonmedical.com/symbols
Български	Речникът на символите е наличен в електронен вид на адрес www.argonmedical.com/symbols
Česky	Slovníček symbolů v elektronické formě je umístěn na www.argonmedical.com/symbols
Dansk	Symbolforklaringen findes elektronisk på www.argonmedical.com/symbols
Deutsch	Das Symbolglossar ist elektronisch verfügbar unter www.argonmedical.com/symbols
Ελληνικά	Το γλωσσάρι των συμβόλων παρέχεται ηλεκτρονικά στη διεύθυνση www.argonmedical.com/symbols
Español	El glosario de símbolos puede consultarse en línea en la dirección www.argonmedical.com/symbols
Eesti	Sümbolite sõnastik on elektrooniliselt saadaval aadressil www.argonmedical.com/symbols
Suomi	Symbolisanasto on sähköisessä muodossa osoitteessa www.argonmedical.com/symbols
Français	Le glossaire des symboles est disponible sous forme électronique à l'adresse www.argonmedical.com/symbols
Magyar	A szimbólumok összefoglalása elektronikusan itt található: www.argonmedical.com/symbols
Italiano	Il glossario dei simboli è disponibile in formato elettronico all'indirizzo www.argonmedical.com/symbols
Lietuvių k.	Simbolių žodynas elektronine forma pateiktas www.argonmedical.com/symbols
Latviešu val.	Simbolu glosārijs ir atrodams elektroniskā veidā vietnē www.argonmedical.com/symbols
Nederlands	Uitleg over de symbolen is elektronisch beschikbaar op www.argonmedical.com/symbols
Norsk	Symbolforklaringen befinner seg elektronisk på www.argonmedical.com/symbols
Polski	Wersja elektroniczna słownika symboli znajduje się na stronie www.argonmedical.com/symbols
Português	O glossário de símbolos está localizado eletronicamente em www.argonmedical.com/symbols
Română	Glosarul simbolurilor este stocat electronic la www.argonmedical.com/symbols
Slovenčina	Glosár symbolov je v elektronickej podobe k dispozícii na stránkach www.argonmedical.com/symbols
Svenska	Symbolordlistan finns elektroniskt på www.argonmedical.com/symbols
Türkçe	Semboller sözlüğü elektronik ortamda www.argonmedical.com/symbols adresinde yer almaktadır

Intended Use/Purpose

The SKATER™ Biliary Drainage Catheters is used for fluid drainage procedures.

Device Description

The SKATER™ Biliary Drainage Catheters are used to drain bile from the bile duct and are placed using the Seldinger technique. This product feature radiopaque marker band that provides clear view of the most proximal drainage hole for accurate placement in the biliary tract.

Indication for use

The product is for biliary drainage

Duration

Up to 12 weeks

Warnings

- This device was designed, tested and manufactured for single use only. Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury. Do not reuse, reprocess or re-sterilize this device.
- Inspect the package integrity before use.
- Do not use if package appear open or if the expiry date has been exceeded.
- During insertion, avoid contact with bone, cartilage, and scar tissue which can damage the catheter tip.

Caution (Locking Pigtail): If the catheter is to be removed in another department, we recommend that these guidance notes accompany the patient's case notes to ensure that the relevant personnel are aware of the presence of a locking catheter. It is also advisable to inform the patient.

Precaution

- The product must only be used by licensed, knowledgeable physician and qualified personnel who are familiar with the technique and who can ensure that patients will be monitored and appropriately treated for expected complications.
- Ensure the drain is secured and the system is intact to prevent dislodgement. Secure with a catheter fixation device, suture, or tape.
- Assess drain insertion site for signs of leakage, redness, or oozing. These signs may indicate an infection or irritation of the surrounding skin.
- Monitor changes in character or volume of fluid or bleeding.
- It is recommended affixing the catheter in a straight line and that any curvature be applied to the connecting tube.
- The drain also provides a pathway for bacteria to get into the wound. The risk of infection can be due to ascending bacterial invasion, foreign body reaction, decreased local tissue resistance, bacterial hiding places, poor placement due to kinked drain, and poor postoperative management.
- It is recommended that the physician follow the hospital's standard of care procedures for drainage catheters.
- As with any draining procedure, there is risk of infection, pain, discomfort, inadvertent tissue damage, and fluid loss. When used as intended by a physician, the benefits of using the SKATER™ Biliary Drainage Catheter outweigh the risks associated with the use of the device.

Preparation

- Advance the straightener carefully over the curvature of the catheter, while straightening the curvature with your fingers.
- **Locking Pigtail** - Gently pull the thread to avoid unintentional looping of the thread.
- Introduce the chosen stiffener (if plastic: activate the hydrophilic coating with saline) and fasten the stiffener onto the catheter (Luer Lock).
- Remove the straightener
- Activate the coating by saline.

Procedure

- Select and prepare the drainage site using standard technique.
- Insert an appropriate guidewire using standard technique.
- Dilate the tract, if necessary.
- Advance the catheter over the guidewire in fluoroscopy.
- When the catheter tip is inside the biliary tree, loosen the metal stiffener and replace it by the hydrophilic plastic stiffener. Advance the catheter over the guidewire as far as duodenum.
- **Non-Locking Pigtail**
 1. Remove the stiffener.
 2. Verify correct position of the catheter using fluoroscopy.
 3. The radiopaque marker indicates the last side hole and must be positioned in the biliary passage.
 4. Remove the guidewire

5. Connect the catheter to a drainage bag using a suitable connecting tube.
- **Locking Pigtail**
 1. Remove the stiffener and the guidewire.
 2. Gently pull the thread turning the catheter counter clockwise to ensure correct position and to complete curvature.
 3. Connect the catheter to a drainage bag using a suitable connecting tube.
 4. Fix the thread tightly by winding it around the slot in the adaptor and finish up with pressing the clip onto the slot and twist off the handle from the clip.
 5. Verify correct position of the catheter using fluoroscopy.
 6. The radiopaque marker indicates the last side hole and must be positioned in the biliary passage.

Removal of catheter

- Disconnect the drainage bag connector tube from catheter.
- **Locking Pigtail**
 1. Loosen the female/male adaptor from the standard luer hub at the catheter to deactivate the valve.
 2. Remove the clip and unwind the suture. Check that both threads are loose and cut one thread in order to loosen the pigtail.
- Pull the catheter out gently. If access is to be maintained, a straight floppy tip guidewire passed through the catheter will facilitate removal while maintaining access.

Disposal

After use, handle and dispose in accordance with hospital policies and procedures concerning biohazard materials and waste.

Storage

Store in controlled room temperature.

NOTE: In the event a serious incident related to this device occurs, the event should be reported to Argon Medical at quality.regulatory@argonmedical.com as well as to the competent health authority where the user/patient resides.

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