ARGON

MEDICAL DEVICES

ARGON MEDICAL DEVICES, INC.
 1445 Flat Creek Road,
 Athens, Texas 75751 USA
 Tel: +1 800-927-4669;
 Tel: +1 903-675-9321
 www.argonmedical.com

Argon Medical Devices UK Ltd
 Eastgate Business Centre
 Eastern Avenue
 Burton-on-Trent
 Staffordshire
 DE13 0AT

EMERGO EUROPE
 Prinsessegracht 20
 2514 AP The Hague
 The Netherlands
 +31 70 345 8570

 <https://www.argonmedical.com/resources/product-information>

Device Description: The Tru-Core™ II Automatic Biopsy Instrument has a 22mm throw with a 19mm sample notch. The device includes a spring loaded hand held unit which houses and fires a needle set under the user guidance of two thumb knobs. It is available in various sizes (14, 16, 18 and 20ga) with lengths varying from 10 cm to 25 cm. The 18gx20cm and 18gx25cm instruments are also available in the URO configuration with the stylet notch upside-down and is specifically for prostate biopsies.

The instrument can be used with a corresponding Co-Axial Introducer Needle (packaged and sold separately).

Indications for use: The Tru-Core™ II Automatic Biopsy Instrument is used to obtain multiple core samples from soft tissue such as liver, kidney, prostate, breast, etc and not intended for bone.

Contraindications: For use only for core biopsies of soft tissue as determined by a licensed physician. The instrument should be used by a physician familiar with the possible side effects, typical findings, limitations, indications and contraindications of core needle biopsy. Physician judgment is required when considering biopsy on patients with bleeding disorder, or receiving anti-coagulant medications.

Side Effect/Risk

- It is not uncommon for patients to experience bleeding, infection at insertion site, fever, pain at the biopsy site, swelling at the biopsy site, adjacent tissue injury and hematoma after the biopsy. When used as intended by a physician, the benefits of using the Tru-Core Biopsy Instrument outweigh the risks associated with the use of the device.

Warnings

- Contents are supplied sterile and are intended for single use only. Do not attempt to clean or resterilize this product.
- Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection and/or other injury.
- Inspect the package integrity before use.
- Do not use if package is open or damaged and if the expiry date has been exceeded.
- Do not continue to use if any of the component are damaged during the procedure.
- Verify integrity of the needle before loading instrument and after firing(s). If needle is damaged, replace the whole system (needle set and instrument).

Caution

- **DO NOT ATTEMPT TO OPEN LID!**
- **Never test fire the Tru-Core™ II Automatic Biopsy Instrument.**

Precaution

- Invasive contact will occur with the patient. Period of contact typically be less than 30 min.
- The device displays a numeric sequence. The user must follow a numeric sequence to operate the Tru-Core™ II Automatic Biopsy Device.
- Tru-Core™ II Biopsy Needles been validated for up to 15 consecutive firings without failure.
- When used repeatedly in the same patient, the device should be inspected for damage or wear after each core sample is taken.

Note : These instructions for the Tru-Core™ II Automatic Biopsy Instrument are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

Pre-Procedural Preparation

1. Slide the cannula knob (1) proximally until the mechanism “clicks” into place (Image 1).

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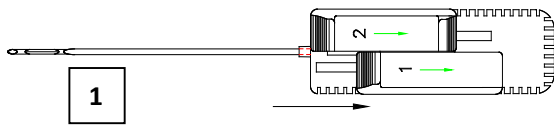
©2017 Argon Medical Devices, Inc. All rights reserved.
 X9585333/0221A


ENGLISH

**Tru-Core™II and Tru-Core™II URO
 Automatic Biopsy Instrument**

Intended Use/Purpose: The Tru-Core II Automatic Biopsy Instrument is a single-use disposable device used for harvesting multiple core specimens from soft tissue.

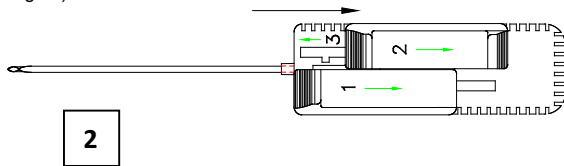
2. Slide the stylet knob (2) proximally until the mechanism “clicks” into place (Image 2).

Directions for Use


3. The device is now ready for use; insert needle proximal to area to be sampled. **Warning:** Be careful not to push stylet thumb knob until needle is inserted to desired location. Verify proper location of needle.

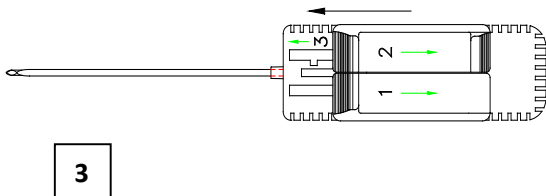4. Slide stylet knob (2) forward until device fires (Image 3).

5. Withdraw the needle from the biopsy site.

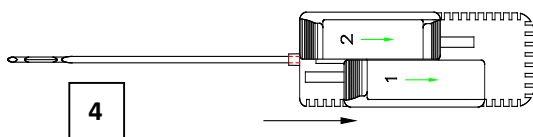6. To remove the biopsy specimen from the needle, engage the cannula knob (1) until the mechanism “clicks” into place (Image 4).

7. If more samples are required, repeat steps 2 through 7.

Disposal: After use, this product may be a potential biohazard. Handle in a manner, which will prevent accidental sharps puncture. Dispose of in accordance with applicable laws and regulations.

Storage: Store at standard ambient temperature.

NOTE: In the event a serious incident related to this device occurs, the event should be reported to Argon Medical at quality.regulatory@argonmedical.com as well as to the competent health authority where the user/patient resides.

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