**ARGON MEDICAL DEVICES, INC.**

1445 Flat Creek Road Athens, Texas 75751 USA
Tel: 800-927-4669;
Tel: +1 903-675-9321
www.argonmedical.com

**EMERGO EUROPE**

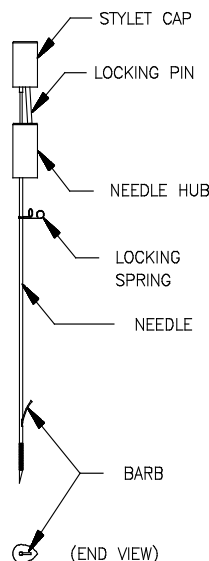
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands
+31 70 345 8570

**Emargo Consulting (UK) Limited**

c/o Cr 360 - UL International
Compass House, Vision Park Histon
Cambridge CB24 9BZ
United Kingdom

<https://www.argonmedical.com/resources/product-information>

**Image Titles:**

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Date of Issue: 07/10/2023 Rev. B

Directions for Use:

The barb is deployed out of the side eyelet which is located 2 cm proximal to the needle point and aligned with the point of the cap.

- Both needles are inserted into the prostate, under guidance, to the appropriate location.
- Each barb is deployed by holding the needle hub and pulling the stylet cap back approximately 1 cm (locking pin snaps up).
- Allow the prostate to settle into its bed.
- While holding the needle hub, gently squeeze the locking spring and slide it down the needle until it firmly contacts the grid. Repeat this procedure for both needles.
- To remove each needle from the prostate, hold the needle hub and push the stylet cap forward until it is fully against the cannula hub. The stylet cap may need to be rotated slightly while pushing forward to release the locking pin. Remove the needle.

Note: These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

Possible Complications:

- Minor Bleeding
- Hematuria
- Infection
- Urinary infections
- Rectal complications
- incidence of urinary incontinence

Disposal:

After use, this product may be a potential biohazard. Handle in a manner which will prevent accident puncture. Dispose in accordance with applicable laws and regulations.

Storage

Store at controlled room temperature.

Note: A notice to the user and/or patient that any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

English	The symbols glossary is located electronically at www.argonmedical.com/symbols
German	Das Glossar der Symbole kann unter www.argonmedical.com/symbols abgerufen werden
French	Le glossaire des symboles est disponible au format électronique à l'adresse www.argonmedical.com/symbols
Spanish	El glosario de símbolos puede consultarse electrónicamente en www.argonmedical.com/symbols
Italian	Il glossario dei simboli è disponibile in formato elettronico all'indirizzo www.argonmedical.com/symbols
Swedish	Symbolförklaringen hittas elektroniskt på www.argonmedical.com/symbols
Portuguese	O glossário de símbolos está localizado eletronicamente no endereço www.argonmedical.com/symbols
Greek	Το γλωσσάρι των συμβόλων παρέχεται ηλεκτρονικά στη διεύθυνση www.argonmedical.com/symbols
Dutch	Uitleg over de symbolen is elektronisch beschikbaar op www.argonmedical.com/symbols
Finnish	Elektroninen symbolisanasto on osoitteessa www.argonmedical.com/symbols
Norwegian	Symbolforklaringen finnes seg elektronisk på www.argonmedical.com/symbols
Danish	Symbolforklaringen findes elektronisk på www.argonmedical.com/symbols

en - English**Prostate Stabilization Set and Prostate Seeding Set****Intended Use/Purpose:**

The Prostate Stabilization and Seeding Set is intended to facilitate transperineal, radioactive seed implantation.

Device Description:

Prostate Stabilization and Prostate Seeding needles are designed with echogenic tip, promoting accurate placement under ultrasound.

Indications for Use: The Prostate Stabilization Set and Prostate Seeding Set are intended to stabilize the prostate and give access for insertion of radioactive isotopes (seeds) into the prostate.

Warnings:

- Contents are supplied sterile and are intended for single use. Do not re-sterilize. Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury.
- Inspect the package integrity before use.
- Do not use if package is open or damaged and if the expiry date has been exceeded.
- Do not continue to use if any of the component are damaged during the procedure.
- Do not advance the needle in dense tissue.
- Failure to use guidance imaging when inserting the needle may cause needle breakage.

Precautions:

- The device is intended for use by qualified licensed physician.

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