ARGON MEDICAL DEVICES, INC.
1445 Flat Creek Road
Athens, Texas 75751 USA
Tel: +1 (903) 675 9321
Tel: +1 (800) 927 4669
www.argonmedical.com

EC REP
EMERGO EUROPE
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands
+31 70 345 8570

UK REP
Emergo Consulting (UK) Limited
c/o Cr 360 - UL International
Compass House, Vision Park Histon
Cambridge CB24 9BZ
United Kingdom

X9585311-0321

Date of Issue: 08/14/2023 Rev. B

 <https://www.argonmedical.com/resources/product-information>

English	The symbols glossary is located electronically at www.argonmedical.com/symbols
Spanish	El glosario de símbolos puede consultarse electrónicamente en www.argonmedical.com/symbols
French	Le glossaire des symboles est disponible au format électronique à l'adresse www.argonmedical.com/symbols
Portuguese	O glossário de símbolos está localizado eletronicamente no endereço www.argonmedical.com/symbols
German	Das Glossar der Symbole kann unter www.argonmedical.com/symbols abgerufen werden
Dutch	Uitleg over de symbolen is elektronisch beschikbaar op www.argonmedical.com/symbols
Italian	Il glossario dei simboli è disponibile in formato elettronico all'indirizzo www.argonmedical.com/symbols
Swedish	Symbolförklaringen hittar elektroniskt på www.argonmedical.com/symbols
Greek	Το γλωσσάρι των συμβόλων παρέχεται ηλεκτρονικά στη διεύθυνση www.argonmedical.com/symbols
Finnish	Elektroninen symbolisanasto on osoitteessa www.argonmedical.com/symbols
Danish	Symbolforklaringen findes elektronisk på www.argonmedical.com/symbols
Norwegian	Symbolforklaringen befinner seg elektronisk på www.argonmedical.com/symbols
Turkish	Semboller sözlüğü elektronik ortamda www.argonmedical.com/symbols adresinde yer almaktadır
Russian	Электронный глоссарий символов расположен по адресу www.argonmedical.com/symbols

ENGLISH

GALACTOGRAPHY KIT

Intended Use/Purpose: Galactography Kit is intended for the purpose of performing Breast Galactography.

Device Description: The Galactography Kit consists of blunt dilators of different sizes and a blunt tipped cannula that can be connected to a Luer slip syringe to inject contrast media.

Indications for Use: Galactography Kit is designed to perform contrast media enhanced radiography of the mammary ducts.

Contraindications: For use only for Breast Galactography as determined by a licensed physician. This kit should be used by a physician familiar with the possible side effects, typical findings, limitations, indications and contraindications of Breast Galactography.

Warnings

- Contents are supplied sterile and are intended for single patient use only. Do not attempt to clean or resterilize.
- Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury.
- Inspect the package integrity before use.

- Do not use if package is open or damaged and if the expiry date has been exceeded.
- Do not continue to use if any of the component are damaged during the procedure.

Precautions

- The device is intended for use by a qualified and licensed physician.
- Contrast media will be injected to the patient and transfer rate is controlled by the user. Risk is minimal and acceptable if used as intended by licensed physician.

Note: These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

Suggested Galactography Procedure:

- Identify the duct:**
Before attempting to perform galactography, first make sure that a discharge can be identified from a single opening on the nipple. Clean the nipple with alcohol. Use of an incandescent lamp to warm the nipple will help to relax the sphincter, thus making cannulation of the duct easier. Use of a dissecting microscope, or a head mounted magnifying lens is helpful.
- Dilate the duct:**
After the nipple has had a chance to relax, manipulate out a very small amount of the discharge to identify the exact orifice to be cannulated. Use a small diameter (000000/.010" or 00000/.012") dilator. Gently rotate the dilator between the index finger and thumb as you insert it into the opening. Dilate with increasingly larger dilators. Once the abnormal duct is dilated, the discharge will usually flow more freely.

RTD001X				DGK002X, DGK002SX	
Color	Size	Color	Size	Color	Size
Blue	- 000000	Turquoise	- 0000	Green	- 00
White	- 000000	Black	- 000	Yellow	- 0
				White	- .012"

- Cannulate the duct:**
Fill the 1cc tuberculin syringe with water soluble contrast (undiluted Conray 60). Attach the blunt tipped cannula to the tuberculin syringe. Remove any air bubbles, as these will appear as filling defects on the galactogram. Gently place the cannula into the duct no deeper than 0.5 to 1cm. Gently inject 0.2 to 0.3cc contrast, or until there is slight resistance or some discomfort. Remove the cannula and obtain mammograms in 2 views with minimal compression. If the duct is not completely filled, or if the diagnosis is not clear, repeat the injection and mammograms. Magnification views can also be helpful.

If extravasation has occurred, the procedure should be discontinued and rescheduled in two to three weeks.

Disposal: After use, this product may be a potential biohazard. Handle in a manner which will prevent accident puncture. Dispose in accordance with applicable laws and regulations.

Storage

Store at controlled room temperature.

NOTE: In the event a serious incident related to this device occurs, the event should be reported to Argon Medical at quality.regulatory@argonmedical.com as well as to the competent health authority where the user/patient resides.

References

Tabar, L., Dean, P. *Interventional radiologic procedures in the investigation of lesions of the breast. Radiol Clin North Am* 1979; 17:607-621.
Tabar, L., Dean, P., Pentek, Z. *Galactography: the diagnostic procedure of choice for nipple discharge. Radiology*, 1983; 149:31-38.

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