ARGON MEDICAL DEVICES, INC.
1445 Flat Creek Road
Athens, Texas 75751 USA
Tel: 800-927-4669;
Tel: +1 903-675-9321
www.argonmedical.com

EC REP **EMERGO EUROPE**
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands
+31 70 345 8570

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UK REP **Emergo Consulting (UK) Limited**
c/o Cr 360 - UL International
Compass House, Vision Park Histon
Cambridge CB24 9BZ
United Kingdom

<https://www.argonmedical.com/resources/product-information>

English	The symbols glossary is located electronically at www.argonmedical.com/symbols
Bulgaria	Речникът на символите е наличен в електронен вид на адрес www.argonmedical.com/symbols
Czech	Slovníček symbolů je elektronicky umístěn na www.argonmedical.com/symbols
Danish	Symbolforklaringen findes elektronisk på www.argonmedical.com/symbols
German	Das Glossar der Symbole kann unter www.argonmedical.com/symbols abgerufen werden
Greek	Το γλῶσσάρι των συμβόλων παρέχεται ηλεκτρονικά στη διεύθυνση www.argonmedical.com/symbols
Spanish	El glosario de símbolos puede consultarse electrónicamente en www.argonmedical.com/symbols
Estonian	Sümbolite seletuste kogu on elektrooniliselt saadaval aadressil www.argonmedical.com/symbols
Finnish	Elektroninen symbolisanasto on osoitteessa www.argonmedical.com/symbols
French	Le glossaire des symboles est disponible au format électronique à l'adresse www.argonmedical.com/symbols
Croatian	Pojmovnik simbola nalazi se u elektroničkom obliku na www.argonmedical.com/symbols
Hungarian	A szimbólumok összefoglalása elektronikusán itt található: www.argonmedical.com/symbols
Italian	Il glossario dei simboli è disponibile in formato elettronico all'indirizzo www.argonmedical.com/symbols
Lithuanian	Simbolių žodyno elektroninė forma pateikta www.argonmedical.com/symbols
Latvian	Simbolu glosārijs ir atrodams elektroniskā veidā vietnē www.argonmedical.com/symbols
Dutch	Uitleg over de symbolen is elektronisch beschikbaar op www.argonmedical.com/symbols
Norwegian	Symbolforklaringen befinner seg elektronisk på www.argonmedical.com/symbols
Polish	Glosariusz symboli w postaci elektronicznej dostępny jest pod adresem www.argonmedical.com/symbols
Portuguese	O glossário de símbolos está localizado eletronicamente no endereço www.argonmedical.com/symbols
Romanian	Glosarul simbolurilor este stocat electronic la www.argonmedical.com/symbols
Slovak	Glosár symbolov je v elektronickej podobe k dispozícii na stránkach www.argonmedical.com/symbols
Slovenian	Slovar simbolov se nahaja v elektronski obliki na www.argonmedical.com/symbols
Swedish	Symbolforklaringen hittas elektroniskt på www.argonmedical.com/symbols
Turkish	Semboller sözlüğü elektronik ortamda www.argonmedical.com/symbols adresinde yer almaktadır
Arabic	متوفر بمركز الرموز الإلكتروني على www.argonmedical.com/symbols

©2013-2017 Argon Medical Devices, Inc. All rights Reserved
SuperCore™ is a trademark of Argon Medical Devices, Inc.
X9585762

Date of Issue: 06/19/2023 Rev. B

ENGLISH

Caution: Federal (USA) Law restricts this device to sale by or on the order of a physician.

SuperCore™ Biopsy Instrument

INTENDED USE/PURPOSE: The SuperCore™ Biopsy Instrument is a semi-automatic biopsy instrument used to obtain soft tissue biopsies.

DEVICE DESCRIPTION: The SuperCore™ Biopsy Instrument is composed of an echogenic spring loaded biopsy needle fitted into a plastic handle permitting single handed specimen collection. The position of the needle may be visualized by x-ray, CT or ultrasound. The needle has numerically ordered centimeter markings to facilitate precise depth placement. The adjustable instrument allows for a throw of 1.2cm or 2.2cm exposing the specimen notch either 19mm or 9.5mm, providing clinical flexibility. The SuperCore™ Biopsy Instrument packaged with a co-axial introducer needle has a supplemental male/female Luer adapter to allow for length adjustment through the coaxial introducer when the instrument is used at the 1.2cm throw.

INDICATIONS FOR USE: Disposable guillotine soft tissue instrument for obtaining percutaneous or surgical histological samples, (e.g. lung, liver, spleen, kidney, prostate, lymph nodes, breast, thyroid, and pancreas).

CONTRAINDICATIONS: These needles should be used by a physician familiar with the possible side effects, typical findings, limitations, indications and contraindications of soft tissue biopsy. Physician judgment is required when considering biopsy on patients with bleeding disorder or receiving anti-coagulant medications.

Side Effect/Risk

- It is not uncommon for patients to experience bleeding, infection at insertion site, fever, pain at the biopsy site, swelling at the biopsy site, adjacent tissue injury and hematoma after the biopsy. When used as intended by a physician, the benefits of using the SuperCore Biopsy Instrument outweigh the risks associated with the use of the device.

WARNINGS

- Contents are supplied sterile and are intended for single use only. Do not reuse, reprocess or re-sterilize this device.
- Reuse or reprocessing has not been evaluated and may lead to its failure and subsequent patient illness, infection or other injury.
- Inspect the package integrity before use.
- Do not use if package is open or damaged and if the expiry date has been exceeded.
- Do not continue to use if any of the component are damaged during the procedure.
- Functional tests are performed during manufacture. Do not try to "dry fire" the device.

PRECAUTIONS


- The device should be used by a qualified, licensed physician.
- Handle in a manner that will prevent accidental puncture.
- This instrument can be used with an appropriate Co-Axial Introducer Needle.

Caution: Other manufacturer's co-axial introducer needles have not been tested for compatibility with the SuperCore™ Biopsy Instrument.


NOTE: These instructions are NOT meant to define or suggest any medical or surgical technique. The individual practitioner is responsible for the proper procedure and techniques to be used with this device.

PREPARATION AND DIRECTION FOR USE


- Optional use of co-axial introducer needle with biopsy instrument: Advance the co-axial introducer needle, using standard technique, to a position proximal to the lesion. Position the adjustable needle stop proximal to the skin. Remove the stylet from the co-axial introducer needle, leaving the cannula in position.
- Cock the SuperCore™ biopsy instrument by pulling back on the plunger as in fig. 1 until a firm click is felt at either the 1.2cm or 2.2cm position. (User selectable.)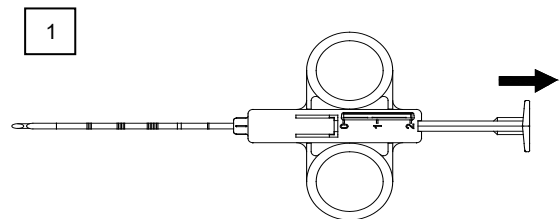


- If using the 1.2cm position with a co-axial introducer needle, install the male/female Luer adapter onto the co-axial cannula as in fig 2. before proceeding.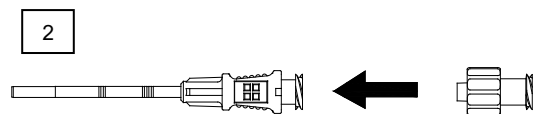


- With stylet fully retracted (specimen notch completely covered by cannula as in fig. 3), insert needle percutaneously or through the co-axial introducer needle proximal to area to be sampled. Warning: Be careful not to press plunger until needle is inserted to desired location. Verify proper location of needle.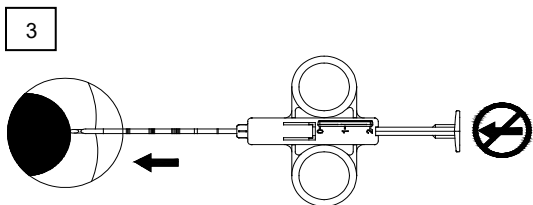

4. Move plunger with thumb to first stop to advance stylet exposing specimen notch (1.2 or 2.2cm). Move needle back and forth a few mm to fill specimen notch. Press plunger past stop with thumb, automatically triggering the cannula to close, trapping the specimen in the sample notch (fig. 4).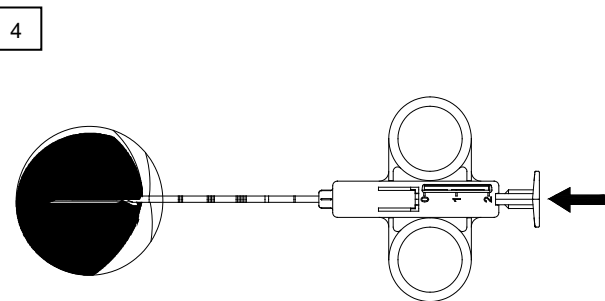

5. Draw needle out. Pull back plunger to 1.2 or 2.2cm position, recharging the spring. Push plunger to first stop exposing the specimen notch. Remove tissue specimen from notch in stylet. Be careful not to push plunger past first stop during this step (fig. 5).

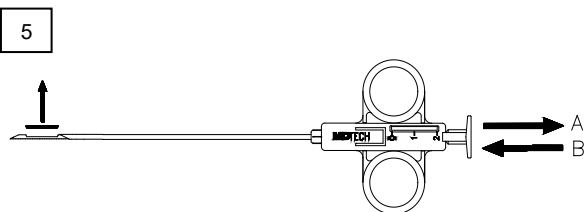DISPOSAL: After use, this product may be a potential biohazard. Handle in a manner, which will prevent accidental sharps puncture. Dispose in accordance with applicable laws and regulations.

STORAGE: Store at standard ambient temperature.

NOTE: In the event a serious incident related to this device occurs, the event should be reported to Argon Medical at quality.regulatory@argonmedical.com as well as to the competent health authority where the user/patient resides.

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