ARGON MEDICAL DEVICES, INC.

1445 Flat Creek Road
Athens, Texas 75751 USA
Tel: +1 (903) 675 9321
Tel: +1 (800) 927 4669
www.argonmedical.com

Emergo Consulting (UK) Limited
c/o Cr 360 - UL International
Compass House, Vision Park Histon
Cambridge CB24 9BZ
United Kingdom

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Intended Use/Purpose

Guidewires are intended for use in angiographic procedure to introduce and position catheters and interventional devices within the peripheral vasculature.

Device Description

Guidewires are designed to facilitate the passage of catheters and sheath introducers into the vasculature utilizing the Seldinger or Modified Seldinger techniques. The device is made with stainless steel and available with or without PTFE coating.

Indications for Use

Guidewires are indicated to facilitate the percutaneous placement of intravascular devices during diagnostic and intervention procedures.

Contraindications for Use

There are no known contraindications. Refer to the device label for any additional product-specific contraindication that may apply.

Possible Complications/Adverse Events

The following complications associated with Guidewire may include, but are not limited to:

- Air embolism
- Hematoma at the puncture site
- Infection
- Perforation of vessel walls
- Perforation of adjacent organs and/or tissues.

Warnings

- This device is designed and intended for ONE-TIME USE ONLY. Do not resterilize and/or reuse.
- Discard after one procedure. Structural integrity and/or function may be impaired through reuse or cleaning.
- Guidewires are extremely difficult to clean after exposure to biological materials and may cause adverse patient reactions if reused.
- Withdrawal, pullback, or manipulation of the guidewire when resistance is met may cause guidewire damage, breakage, or embolization.
- Avoid withdrawing guidewire through metal needles, Guidewire may shear, or PTFE coating may scrape off against the needle bevel.
- Do not excessively torque the guidewire.
- Do not entrap or over-rotate the distal tip of the guidewire, it can lead to wire rupture.
- Do not excessively bend the guidewire, it may result in wire fracture.

Precautions

- Contents sterile unless package is damaged or opened.
- Inspect the package integrity before use.
- Guidewires are intended to be used by trained medical personnel in a clinical setting.
- The provided dispenser is best means of storage and handling of the guidewire, avoid coiling guidewires in less than 8" diameter during use.
- Guidewires should be routinely inspected prior to use and discarded should any deformities be present in the guidewire.
- Use prior to the "Exp" date.
- If strong resistance is met during manipulation, discontinue the procedure and determine the cause for the resistance before proceeding.
- Avoid bending, kinking or modifying the shape of the wire.
- Guidewire placement should be monitored during placement and manipulation using fluoroscopic or suitable imaging method.
- Sufficient Guidewire length must remain exposed to maintain suitable grip proximally at all times.
- PTFE guidewires should not be sterilized by irradiation sterilization.
- Twisting or entanglement of the guidewires may occur when more than one wire is being used simultaneously. This can be prevented by carefully isolating and marking the proximal end of the wires.

CAUTIONS for Movable Core Guidewires

While the guidewire is in a vessel, do not advance the movable core if the tip is in a curved shape. Never twist or force the core because excessive force may cause it to penetrate the coil and damage the vessel.

Preparation

1. Make sure the product is compatible with other equipment.

Note: If unable to remove guide wire from dispenser, separate straightener from hoop dispenser.

2. Flush the guidewire with sterile heparinized normal saline or a similar isotonic solution by connecting a syringe to the luer hub of the guidewire dispenser.
3. Inject saline with syringe in a vertical position.

Direction for Use

4. Insert the flexible end of the guidewire into the needle. Avoid overly vigorous manipulation of the guidewire to prevent damage of vessel or device. If "J" wire is used, prepare "J" tip by sliding J-Straightener over tip to straighten. Insert J-tip inside J-Straightener into the needle hub gently.
5. Advance the guidewire through the needle.
6. Position the guidewire.
7. Hold the guidewire in place and withdraw the needle.
8. Advance the intravascular device or catheter over the guidewire.

Movable Guidewires

To lengthen the flexible segment of a movable-core guidewire, gently withdraw the core. Do not advance movable-core within vasculature.

Disposal

After use, handle and dispose in accordance with end-user facility's policies and procedures concerning biohazardous materials and waste.

Storage

Store at controlled room temperature.

NOTE: In the event a serious incident related to this device occurs, the event should be reported to Argon Medical at quality.regulatory@argonmedical.com as well as to the competent health authority where the user/patient resides.

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